



MAGLUMI® Anti-GAD (CLIA)

INTENDED USE

The kit is an *in vitro* chemiluminescence immunoassay for the quantitative determination of Glutamic acid decarboxylase antibody (Anti-GAD) in human serum and plasma using the MAGLUMI series Fully-auto chemiluminescence immunoassay analyzer and Biolumi series Integrated System, and the assay is used for an aid in the diagnosis of individuals with suspected or confirmed diabetes mellitus type 1.

SUMMARY

Glutamic acid decarboxylase (GAD) is an important component of the 64-kD islet autoantigen. GAD is present in all human islet cells (e.g. α , β , δ) and there are two forms of GAD, termed GAD65 and GAD67 that are 76% homologous in amino acid sequence¹. Most type 1A diabetic sera only target GAD65 protein epitopes dependent on conformation and do not react with denatured GAD65. The two major epitopes targeted by diabetic sera are in the middle part of GAD65 between amino acids 240–235 (termed IDDM-E1) and at the C-terminal region of GAD65 between amino acids 451–570 (termed IDDM-E2)². GAD is a major target of the autoimmune response that occurs in type 1 diabetes mellitus³.

The primary ways to assess the activation of anti-islet autoimmunity are to measure autoantibodies to insulin in the NOD mouse and autoantibodies to insulin, GAD65, or IA-2 in humans². Autoantibodies to GAD65 are observed months to years before the clinical onset of diabetes and are present in the sera of 70–80% of patients with T1D⁴. Some results confirm high type 1 diabetes risk among individuals with multiple antibodies to the three major diabetes autoantigens (GAD, IA2/ICA512, and insulin) and show that this strategy can successfully predict type 1 diabetes among schoolchildren⁵.

TEST PRINCIPLE

Sandwich chemiluminescence immunoassay.

The sample, ABEI labeled with GAD65 antigen, buffer, magnetic microbeads coated with GAD65 antigen are mixed thoroughly and incubated, reacting to form sandwich complexes. After precipitation in a magnetic field, the supernatant is decanted and then a wash cycle is performed. Subsequently, the Starter 1+2 are added to initiate a chemiluminescent reaction. The light signal is measured by a photomultiplier as relative light units (RLUs), which is proportional to the concentration of Anti-GAD present in the sample.

REAGENTS

Kit Contents

Component	Description	100 tests/kit	50 tests/kit	30 tests/kit
Magnetic Microbeads	Magnetic microbeads coated with GAD65 antigen (~10.7 µg/mL) in HEPES buffer, NaN ₃ (<0.1%).	2.5 mL	1.5 mL	1.0 mL
Calibrator Low	A low concentration of Anti-GAD in Tris-HCl buffer, NaN ₃ (<0.1%).	1.0 mL	1.0 mL	1.0 mL
Calibrator High	A high concentration of Anti-GAD in Tris-HCl buffer, NaN ₃ (<0.1%).	1.0 mL	1.0 mL	1.0 mL
Buffer	Tris-HCl buffer, NaN ₃ (<0.1%).	5.5 mL	3.5 mL	2.7 mL
ABEI Label	ABEI labeled with GAD65 antigen (~0.417 µg/mL) in PBS buffer, NaN ₃ (<0.1%).	12.5 mL	7.0 mL	4.8 mL
Control 1	A low concentration of Anti-GAD (20.0 IU/mL) in Tris-HCl buffer, NaN ₃ (<0.1%).	1.0 mL	1.0 mL	1.0 mL
Control 2	A high concentration of Anti-GAD (200 IU/mL) in Tris-HCl buffer, NaN ₃ (<0.1%).	1.0 mL	1.0 mL	1.0 mL
All reagents are provided ready-to-use.				

Warnings and Precautions

- For *in vitro* diagnostic use.
- For professional use only.
- Exercise the normal precautions required for handling all laboratory reagents.
- Personal protective measures should be taken to prevent any part of the human body from contacting samples, reagents, and controls, and should comply with local operating requirements for the assay.
- A skillful technique and strict adherence to the package insert are necessary to obtain reliable results.
- Do not use kit beyond the expiration date indicated on the label.
- Do not interchange reagent components from different reagents or lots.
- Avoid foam formation in all reagents and sample types (specimens, calibrators and controls).
- All waste associated with biological samples, biological reagents and disposable materials used for the assay should be considered potentially infectious and should be disposed of in accordance with local guidelines.
- This product contains sodium azide. Sodium azide may react with lead or copper plumbing to form highly explosive metal azides. Immediately after disposal, flush with a large volume of water to prevent azide build-up. For additional information, see Safety Data Sheets available for professional user on request.

Note: If any serious incident has occurred in relation to the device, please report to Shenzhen New Industries Biomedical Engineering Co., Ltd. (Snibe) or our authorized representative and the competent authority of the Member State in which you are established.

Reagent Handling

- To avoid contamination, wear clean gloves when operating with a reagent kit and sample. When handling reagent kit, replace the gloves that have been in contact with samples, since introduction of samples will result in unreliable results.
- Do not use kit in malfunction conditions; e.g., the kit leaking at the sealing film or elsewhere, obviously turbid or precipitation is found in reagents (except for Magnetic Microbeads) or control value is out of the specified range repeatedly. When kit in malfunction conditions, please contact Snibe or our authorized distributor.
- To avoid evaporation of the liquid in the opened reagent kits in refrigerator, it is recommended that the opened reagent kits to be sealed with reagent seals contained within the packaging. The reagent seals are single use, and if more seals are needed, please contact Snibe or our authorized distributor.
- Over time, residual liquids may dry on the septum surface. These are typically dried salts and have no effect on assay efficacy.
- Use always the same analyzer for an opened reagent integral.
- For magnetic microbeads mixing instructions, refer to the Preparation of the Reagent section of this package insert.
- For further information about the reagent handing during system operation, please refer to Analyzer Operating Instructions.

Storage and Stability

- Do not freeze the integral reagents.
- Store the reagent kit upright to ensure complete availability of the magnetic microbeads.
- Protect from direct sunlight.

Stability of the Reagents	
Unopened at 2-8°C	until the stated expiration date
Opened at 2-8°C	6 weeks
On-board	4 weeks

Stability of Controls	
Unopened at 2-8°C	until the stated expiration date
Opened at 10-30°C	6 hours
Opened at 2-8°C	6 weeks



0123



130255005M: 100 tests/kit
130655005M: 50 tests/kit
130755005M: 30 tests/kit

Frozen at -20°C	3 months
Frozen and thawed cycles	no more than 3 times

SPECIMEN COLLECTION AND PREPARATION

Specimen Types

Only the specimens listed below were tested and found acceptable.

Specimen Types	Collection Tubes
Serum	Tubes without additive/accessory, or tubes containing clot activator or clot activator with gel.
Plasma	K2-EDTA

- The sample types listed were tested with a selection of sample collection tubes that were commercially available at the time of testing, i.e. not all available tubes of all manufacturers were tested. Sample collection systems from various manufacturers may contain differing materials which could affect the test results in some cases. Follow tube manufacturers' instructions carefully when using collection tubes.

Specimen Conditions

- Do not use grossly hemolyzed/hyperlipidaemia specimens and specimens with obvious microbial contamination.
- Ensure that complete clot formation in serum specimens has taken place prior to centrifugation. Some serum specimens, especially those from patients receiving anticoagulant or thrombolytic therapy, may exhibit increased clotting time. If the serum specimen is centrifuged before a complete clotting, the presence of fibrin may cause erroneous results.
- Samples must be free of fibrin and other particulate matter.
- To prevent cross contamination, use of disposable pipettes or pipette tips are recommended.

Preparation for Analysis

- Inspect all specimens for foam. Remove foam with an applicator stick before analysis. Use a new applicator stick for each specimen to prevent cross contamination.
- Frozen specimens must be completely thawed before mixing. Mix thawed specimens thoroughly by low speed vortexing or by gently inverting. Visually inspect the specimens. If layering or stratification is observed, mix until specimens are visibly homogeneous. If specimens are not mixed thoroughly, inconsistent results may be obtained.
- Specimens should be free of fibrin, red blood cells, or other particulate matter. Such specimens may give reliable results and must be centrifuged prior to testing. Transfer clarified specimen to a sample cup or secondary tube for testing. For centrifuged specimens with a lipid layer, transfer only the clarified specimen and not the lipemic material.
- The sample volume required for a single determination of this assay is 50 µL.

Specimen Storage

Specimens removed from the separator, red blood cells or clot may be stored up to 24 hours at 10-30°C or 14 days at 2-8°C, or 3 months frozen at -20°C. Frozen specimens subjected to up to 2 freeze/thaw cycles have been evaluated.

Specimen Shipping

- Package and label specimens in compliance with applicable local regulations covering the transport of clinical specimens and infectious substances.
- Do not exceed the storage limitations listed above.

Specimen Dilution

- Samples, with Anti-GAD concentrations above the analytical measuring interval, can be diluted with manual dilution procedure. The recommended dilution ratio is 1:100. The concentration of the diluted sample must be >20 IU/mL.
- For manual dilution, multiply the result by the dilution factor.
- Please choose applicable diluents or ask Snibe for advice before manual dilution.

PROCEDURE

Materials Provided

Anti-GAD (CLIA) assay, control barcode labels.

Materials Required (But Not Provided)

- General laboratory equipment.
- Fully-auto chemiluminescence immunoassay analyzer Maglumi 600, Maglumi 800, Maglumi 1000, Maglumi 2000, Maglumi 2000 Plus, Maglumi 4000, Maglumi 4000 Plus, MAGLUMI X8, MAGLUMI X3, MAGLUMI X6 or Integrated System Biolumi 8000, Biolumi CX8.
- Additional accessories of test required for the above analyzers include Reaction Module, Starter 1+2, Wash Concentrate, Light Check, Tip, and Reaction Cup. Specific accessories and accessories' specification for each model refer to corresponding Analyzer Operating Instructions.
- Please use accessories specified by Snibe to ensure the reliability of the test results.

Assay Procedure

Preparation of the Reagent

- Take the reagent kit out of the box and visually inspect the integral vials for leaking at the sealing film or elsewhere. If there is no leakage, please tear off the sealing film carefully.
- Open the reagent area door; hold the reagent handle to get the RFID label close to the RFID reader (for about 2s); the buzzer will beep; one beep sound indicates successful sensing.
- Keeping the reagent straight insert to the bottom along the blank reagent track.
- Observe whether the reagent information is displayed successfully in the software interface, otherwise repeat the above two steps.
- Resuspension of the magnetic microbeads takes place automatically when the kit is loaded successfully, ensuring the magnetic microbeads are totally resuspended homogenous prior to use.

Assay Calibration

- Select the assay to be calibrated and execute calibration operation in reagent area interface. For specific information on ordering calibrations, refer to the calibration section of Analyzer Operating Instructions.
- Execute recalibration according to the calibration interval required in this package insert.

Quality Control

- When new lot used, check or edit the quality control information.
- Scan the control barcode, choose corresponding quality control information and execute testing. For specific information on ordering quality controls, refer to the quality control section of the Analyzer Operating Instructions.

Sample Testing

After successfully loading the sample, select the sample in interface and edit the assay for the sample to be tested and execute testing. For specific information on ordering patient specimens, refer to the sample ordering section of the Analyzer Operating Instructions.

To ensure proper test performance, strictly adhere to Analyzer Operating Instructions.

Calibration

Traceability: This method has been standardized against the WHO International Standard 97/550.

Test of assay specific calibrators allows the detected relative light unit (RLU) values to adjust the master curve.

Recalibration is recommended as follows:

- Whenever a new lot of Reagent or Starter 1+2 is used.
- Every 28 days.
- The analyzer has been serviced.
- Control values lie outside the specified range.

Quality Control

Controls are recommended for the determination of quality control requirements for this assay and should be run in singlicate to monitor the assay performance. Refer to published guidelines for general quality control recommendations, for example Clinical and Laboratory Standards Institute (CLSI) Guideline C24 or other published guidelines⁶.

Quality control is recommended once per day of use, or in accordance with local regulations or accreditation requirements and your laboratory's quality control

procedures, quality control could be performed by running the Anti-GAD assay:

- Whenever the kit is calibrated.
- Whenever a new lot of Starter 1+2 or Wash Concentrate is used.

Controls are only applicable with MAGLUMI and Biolumi systems and only used matching with the same top seven LOT numbers of corresponding reagents. For each target value and range refer to the label.

The performance of other controls should be evaluated for compatibility with this assay before they are used. Appropriate value ranges should be established for all quality control materials used.

Control values must lie within the specified range, whenever one of the controls lies outside the specified range, calibration should be repeated and controls retested. If control values lie repeatedly outside the predefined ranges after successful calibration, patient results must not be reported and take the following actions:

- Verify that the materials are not expired.
- Verify that required maintenance was performed.
- Verify that the assay was performed according to the package insert.
- If necessary, contact Snibe or our authorized distributors for assistance.

If the controls in kit are not enough for use, please order Anti-GAD (CLIA) Controls (REF: 160201470MT) from Snibe or our authorized distributors for more.

RESULTS

Calculation

The analyzer automatically calculates the Anti-GAD concentration in each sample by means of a calibration curve which is generated by a 2-point calibration master curve procedure. The results are expressed in IU/mL. For further information please refer to the Analyzer Operating Instructions.

Interpretation of Results

The optimal cut-off of the Anti-GAD assay was obtained by testing 172 confirmed Type 1diabetes patients, and 148 apparently healthy individuals.

- Samples with Anti-GAD concentration <10 IU/mL should be considered negative.
- Samples with Anti-GAD concentration ≥10 IU/mL should be considered positive.

Results may differ between laboratories due to variations in population and test method. It is recommended that each laboratory establish its own reference interval.

LIMITATIONS

- Results should be used in conjunction with patient's medical history, clinical examination and other findings.
- If the Anti-GAD results are inconsistent with clinical evidence, additional testing is needed to confirm the result.
- Heterophilic antibodies in human serum can react with reagent immunoglobulins, interfering with *in vitro* immunoassays. Patients routinely exposed to animals or animal serum products can be prone to this interference and anomalous values may be observed’.
- Bacterial contamination of the specimens may affect the test results.

SPECIFIC PERFORMANCE CHARACTERISTICS

Representative performance data are provided in this section. Results obtained in individual laboratories may vary.

Precision

Precision was determined using the assay, samples and controls in a protocol (EP05-A3) of the CLSI (Clinical and Laboratory Standards Institute): duplicates at two independent runs per day for 5 days at three different sites using three lots of reagent kits (n = 180). The following results were obtained:

Sample	Mean (IU/mL) (n=180)	Within-Run		Between-Run		Reproducibility	
		SD (IU/mL)	%CV	SD (IU/mL)	%CV	SD (IU/mL)	%CV
Serum Pool 1	10.202	0.319	3.13	0.135	1.32	0.433	4.24
Serum Pool 2	198.232	5.246	2.65	1.808	0.91	7.665	3.87
Serum Pool 3	777.604	12.636	1.62	4.401	0.57	22.951	2.95
Plasma Pool 1	10.071	0.292	2.90	0.074	0.73	0.420	4.17
Plasma Pool 2	196.254	4.686	2.39	1.993	1.02	7.659	3.90
Plasma Pool 3	799.889	12.552	1.57	4.002	0.50	16.471	2.06
Control 1	20.052	0.580	2.89	0.187	0.93	0.938	4.68
Control 2	202.240	4.831	2.39	4.075	2.01	7.598	3.76

Linear Range

5.00-2000 IU/mL (defined by the Limit of Quantitation and the maximum of the master curve).

Reportable Interval

2.00-200000 IU/mL (defined by the Limit of Detection and the maximum of the master curve×Recommended Dilution Ratio).

Analytical Sensitivity

Limit of Blank (LoB) =0.200 IU/mL.

Limit of Detection (LoD) ≈2.00 IU/mL.

Limit of Quantitation (LoQ) ≈5.00 IU/mL.

Analytical Specificity

Interference

Interference was determined using the assay, three samples containing different concentrations of analyte were spiked with potential endogenous and exogenous interferences in a protocol (EP7-A2) of the CLSI. The measurement deviation of the interference substance is within ±10%. The following results were obtained:

Interference	No interference up to	Interference	No interference up to
Hemoglobin	1500 mg/dL	ANA	398 AU/mL
Intralipid	1500 mg/dL	Biotin	0.5 mg/dL
Bilirubin	60 mg/dL	K2-EDTA	22.75 µmol/mL
HAMA	40 ng/mL	Total protein	12 g/dL
Rheumatoid factor	1500 IU/mL		

Cross-Reactivity

Cross-reactivity was determined using the assay, three samples containing different concentrations of analyte were spiked with potential cross-reactant in a protocol (EP7-A2) of the CLSI. The measurement deviation of the interference substance is within ±10%. The following results were obtained:

Cross-reactant	No interference up to	Cross-reactant	No interference up to
GAD67 antibody	2000 IU/mL	Insulin antibody	1500 AU/mL
Tyrosine phosphatase antibody	2000 IU/mL	Thyroid peroxidase antibody	20000 IU/mL
Thyrotrophin receptor antibody	150 IU/L	dsDNA antibody	800 IU/mL

High-Dose Hook

No high-dose hook effect was seen for Anti-GAD concentrations up to 2000000 IU/mL.

Method Comparison

A comparison of the Anti-GAD assay with a commercially available immunoassay, gave the following correlations (IU/mL):

Number of samples measured: 116

Passing-Bablok: $\hat{y}$ =0.9986x+0.0652, τ =0.976.

The clinical specimen concentrations were between 5.177 and 1983.822 IU/mL.

REFERENCES

1. Miao D, Yu L, Eisenbarth G S. Role of autoantibodies in type 1 diabetes [J]. Frontiers in Bioscience A Journal & Virtual Library, 2007, 12(1):1889-1898.
2. Gianani R, Eisenbarth G S. The stages of Type 1A diabetes: 2005[J]. Immunological Reviews, 2005, 204(1):232-249.
3. Wherrett D K, Bundy B, Becker D J, et al. Antigen-based therapy with glutamic acid decarboxylase (GAD) vaccine in patients with recent-onset type 1 diabetes: a randomised double-blind trial[J]. Lancet, 2011, 378(9788):319-327.
4. Min, Li, Lu-Jun, et al. Advances in the cellular immunological pathogenesis of type 1 diabetes[J]. Journal of Cellular and Molecular Medicine, 2014, 470 Anti-GAD-en-EU, V1.0, 2021-12

- 18(5):749-758.
5. Lagasse J M, Brantley M S, Leech N J, et al. Successful Prospective Prediction of Type 1 Diabetes in Schoolchildren Through Multiple Defined Autoantibodies[J]. Diabetes Care, 2002, 25(3):505-511.
 6. CLSI. Statistical Quality Control for Quantitative Measurement Procedures: Principles and Definitions. 4th ed. CLSI guideline C24. Wayne, PA: Clinical and Laboratory Standards Institute; 2016.
 7. Boscato L M, Stuart M C. Heterophilic antibodies: a problem for all immunoassays. Clin Chem 1988;34(1):27-33.



Consult instructions for use



Temperature limit
(Store at 2-8°C)



Contains sufficient for<n> tests



This way up



In vitro diagnostic medical device



Catalogue number



CE marking with notified body ID number



Manufacturer



Use-by date



Keep away from sunlight



Authorized representative in the European Community



Kit component



Batch code

MAGLUMI® and Biolumi® are trademarks of Snibe. All other product names and trademarks are the property of their respective owners.

Summary of safety and performance is available at Eudamed.



Shenzhen New Industries Biomedical Engineering Co., Ltd.
No.23, Jinxiu East Road, Pingshan District, 518122 Shenzhen, P.R. China
Tel: +86-755-21536601 Fax:+86-755-28292740



Shanghai International Holding Corp. GmbH (Europe)
Eiffestrasse 80, 20537 Hamburg, Germany
Tel: +49-40-2513175 Fax: +49-40-255726