



0123



130255001M: 100 tests/kit

130655001M: 50 tests/kit

130755001M: 30 tests/kit

MAGLUMI® C-Peptide (CLIA)

INTENDED USE

The kit is an *in vitro* chemiluminescence immunoassay for the quantitative determination of C-Peptide (C-P) in human serum, plasma and urine using the MAGLUMI series Fully-auto chemiluminescence immunoassay analyzer and Biolumi series Integrated System, and the assay is used for an aid in the diagnosis and treatment of patients with abnormal insulin secretion, including diabetes mellitus.

SUMMARY

C-Peptide is a natural cleavage product of proinsulin released from pancreatic beta cells in equimolar amounts with insulin¹. C-Peptide is a widely used measure of pancreatic beta cell function. It is produced in equimolar amounts to endogenous insulin but is excreted at a more constant rate over a longer time². C-Peptide levels are associated with diabetes type and duration of disease. Specifically a C-Peptide level of less than 0.2 nmol/l is associated with a diagnosis of type 1 diabetes mellitus (T1DM). C-Peptide level may correlate with microvascular and macrovascular complications and future use of insulin therapy, as well as likely response to other individual therapies. In insulin-treated patients with diabetes, measurement of C-Peptide also avoids the pitfall of cross-reaction of assay between exogenous and endogenous insulin². The evaluation of postprandial hypoglycemia should include measurements of plasma glucose, insulin, C-Peptide, proinsulin, and β -hydroxybutyrate collected during an episode in which the patient is experiencing symptoms of hypoglycemia³. The American Diabetes Association (ADA) and the American Association for Clinical Chemistry (AACC) guideline recommend C-Peptide measurements may help distinguish type 1 from type 2 diabetes in ambiguous cases, such as patients who have a type 2 phenotype but present in ketoacidosis⁴. The levels of insulin and C-Peptide and the presence or absence of immune markers in addition to the clinical presentation may help establish the correct diagnosis to distinguish between T1D and T2D in children or adults and determine appropriate treatment⁵. Patients with renal malfunction show elevated C-peptide values, food intake or therapy with β -cell stimulating drugs (e.g. corticosteroids) increase C-peptide secretion. Fasting as well as β -cell inhibiting substances such as insulin or α -sympathomimetic drugs decrease C-peptide levels⁵. Patients with untreated Addison's disease show subnormal C-peptide concentrations.

TEST PRINCIPLE

Sandwich chemiluminescence immunoassay.

The sample magnetic microbeads coated with anti-C-P antibody, ABEI labeled with another anti-C-P antibody, buffer are mixed thoroughly and incubated, forming sandwich complexes. After precipitation in a magnetic field, the supernatant is decanted and then perform a wash cycle. Subsequently, the Starter 1+2 are added to initiate a chemiluminescent reaction. The light signal is measured by a photomultiplier as relative light units (RLUs), which is proportional to the concentration of C-P present in the sample.

REAGENTS

Kit Contents

Component	Description	100 tests/kit	50 tests/kit	30 tests/kit
Magnetic Microbeads	Magnetic microbeads coated with anti-C-P antibody (~10.0 µg/mL) in PBS buffer, NaNa ₃ (<0.1%).	2.5 mL	1.5 mL	1.0 mL
Calibrator Low	A low concentration of C-P antigen in PBS buffer, NaNa ₃ (<0.1%).	1.0 mL	1.0 mL	1.0 mL
Calibrator High	A high concentration of C-P antigen in PBS buffer, NaNa ₃ (<0.1%).	1.0 mL	1.0 mL	1.0 mL
Buffer	PBS buffer, NaNa ₃ (<0.1%).	5.5 mL	3.5 mL	2.7 mL
ABEI Label	ABEI labeled with anti-C-P antibody (~0.500 µg/mL) in PBS buffer, NaNa ₃ (<0.1%).	8.5 mL	5.0 mL	3.6 mL
Control 1	A low concentration of C-P antigen (4.00 ng/mL) in PBS buffer, NaNa ₃ (<0.1%).	1.0 mL	1.0 mL	1.0 mL
Control 2	A high concentration of C-P antigen (10.0 ng/mL) in PBS buffer, NaNa ₃ (<0.1%).	1.0 mL	1.0 mL	1.0 mL
All reagents are provided ready-to-use.				

The control barcode labels are provided.

Warnings and Precautions

- For *in vitro* diagnostic use.
- For professional use only.
- Exercise the normal precautions required for handling all laboratory reagents.
- Personal protective measures should be taken to prevent any part of the human body from contacting samples, reagents, and controls, and should comply with local operating requirements for the assay.
- A skillful technique and strict adherence to the package insert are necessary to obtain reliable results.
- Do not use kit beyond the expiration date indicated on the label.
- Do not interchange reagent components from different reagents or lots.
- Avoid foam formation in all reagents and sample types (specimens, calibrators and controls).
- All waste associated with biological samples, biological reagents and disposable materials used for the assay should be considered potentially infectious and should be disposed of in accordance with local guidelines.
- This product contains sodium azide. Sodium azide may react with lead or copper plumbing to form highly explosive metal azides. Immediately after disposal, flush with a large volume of water to prevent azide build-up. For additional information, see Safety Data Sheets available for professional user on request.

Note: If any serious incident has occurred in relation to the device, please report to Shenzhen New Industries Biomedical Engineering Co., Ltd. (Snibe) or our authorized representative and the competent authority of the Member State in which you are established.

Reagent Handling

- To avoid contamination, wear clean gloves when operating with a reagent kit and sample. When handling reagent kit, replace the gloves that have been in contact with samples, since introduction of samples will result in unreliable results.
- Do not use kit in malfunction conditions; e.g., the kit leaking at the sealing film or elsewhere, obviously turbid or precipitation is found in reagents (except for Magnetic Microbeads) or control value is out of the specified range repeatedly. When kit in malfunction conditions, please contact Snibe or our authorized distributor.
- To avoid evaporation of the liquid in the opened reagent kits in refrigerator, it is recommended that the opened reagent kits to be sealed with reagent seals contained within the packaging. The reagent seals are single use, and if more seals are needed, please contact Snibe or our authorized distributor.
- Over time, residual liquids may dry on the septum surface. These are typically dried salts and have no effect on assay efficacy.
- Use always the same analyzer for an opened reagent integral.
- For magnetic microbeads mixing instructions, refer to the Preparation of the Reagent section of this package insert.
- For further information about the reagent handling during system operation, please refer to Analyzer Operating Instructions.

Storage and Stability

- Do not freeze the integral reagents.
- Store the reagent kit upright to ensure complete availability of the magnetic microbeads.
- Protect from direct sunlight.

Stability of the Reagents	
Unopened at 2-8°C	until the stated expiration date

Opened at 2-8°C	6 weeks
On-board	4 weeks

Stability of Controls	
Unopened at 2-8°C	until the stated expiration date
Opened at 10-30°C	6 hours
Opened at 2-8°C	6 weeks
Frozen at -20°C	3 months
Frozen and thawed cycles	no more than 3 times

SPECIMEN COLLECTION AND PREPARATION

Specimen Types

Only the specimens listed below were tested and found acceptable.

Specimen Types	Collection Tubes
Serum	Tubes without additive/accessory, or tubes containing clot activator or clot activator with gel.
Plasma	K2-EDTA
Urine	/

- The sample types listed were tested with a selection of sample collection tubes that were commercially available at the time of testing, i.e. not all available tubes of all manufacturers were tested. Sample collection systems from various manufacturers may contain differing materials which could affect the test results in some cases. Follow tube manufacturers' instructions carefully when using collection tubes.

Specimen Conditions

- Do not use grossly hemolyzed/hyperlipidaemia specimens and specimens with obvious microbial contamination.
- Ensure that complete clot formation in serum specimens has taken place prior to centrifugation. Some serum specimens, especially those from patients receiving anticoagulant or thrombolytic therapy, may exhibit increased clotting time. If the serum specimen is centrifuged before a complete clotting, the presence of fibrin may cause erroneous results.
- Samples must be free of fibrin and other particulate matter.
- To prevent cross contamination, use of disposable pipettes or pipette tips is recommended.

Preparation for Analysis

- Inspect all specimens for foam. Remove foam with an applicator stick before analysis. Use a new applicator stick for each specimen to prevent cross contamination.
- Frozen specimens must be completely thawed before mixing. Mix thawed specimens thoroughly by low speed vortexing or by gently inverting. Visually inspect the specimens. If layering or stratification is observed, mix until specimens are visibly homogeneous. If specimens are not mixed thoroughly, inconsistent results may be obtained.
- Specimens should be free of fibrin, red blood cells, or other particulate matter. Such Specimens may give reliable results and must be centrifuged prior to testing. Transfer clarified specimen to a sample cup or secondary tube for testing. For centrifuged specimens with a lipid layer, transfer only the clarified specimen and not the lipemic material.
- The sample volume required for a single determination of this assay is 30 µL.
- 24-hours urine, 1:10 prediluted with normal saline.

Specimen Storage

Stability of the serum, plasma and 24-hours urine samples: specimens may be stored up to 24 hours at 10-30°C, or 2 days at 2-8°C, or 3 months frozen at -20°C. Frozen specimens subjected to up to 3 freeze/thaw cycles have been evaluated.

Specimen Shipping

- Package and label specimens in compliance with applicable local regulations covering the transport of clinical specimens and infectious substances.
- Do not exceed the storage limitations listed above.

Specimen Dilution

- Samples, C-P concentrations above the analytical measuring interval, can be diluted with manual dilution procedure. The recommended dilution ratio is 1:10. The concentration of the diluted sample must be >4.00 ng/mL.
- For manual dilution, multiply the result by the dilution factor.
- Please choose applicable diluents or ask Snibe for advice before manual dilution.

PROCEDURE

Materials Provided

C-Peptide (CLIA) assay, control barcode labels.

Materials Required (But Not Provided)

- General laboratory equipment.
- Fully-auto chemiluminescence immunoassay analyzer Maglumi 600, Maglumi 800, Maglumi 1000, Maglumi 2000, Maglumi 2000 Plus, Maglumi 4000, Maglumi 4000 Plus, MAGLUMI X8, MAGLUMI X3, MAGLUMI X6, or Integrated System Biolumi 8000 and Biolumi CX8.
- Additional accessories of test required for the above analyzers include Reaction Module, Starter 1+2, Wash Concentrate, Light Check, Tip, and Reaction Cup. Specific accessories and accessories' specification for each model refer to corresponding Analyzer Operating Instructions.
- Please use accessories specified by Snibe to ensure the reliability of the test results.

Assay Procedure

Preparation of the Reagent

- Take the reagent kit out of the box and visually inspect the integral vials for leaking at the sealing film or elsewhere. If there is no leakage, please tear off the sealing film carefully.
- Open the reagent area door; hold the reagent handle to get the RFID label close to the RFID reader (for about 2s); the buzzer will beep; one beep sound indicates successful sensing.
- Keeping the reagent straight insert to the bottom along the blank reagent track.
- Observe whether the reagent information is displayed successfully in the software interface, otherwise repeat the above two steps.
- Resuspension of the magnetic microbeads takes place automatically when the kit is loaded successfully, ensuring the magnetic microbeads are totally resuspended homogenous prior to use.

Assay Calibration

- Select the assay to be calibrated and execute calibration operation in reagent area interface. For specific information on ordering calibrations, refer to the calibration section of Analyzer Operating Instructions.
- Execute recalibration according to the calibration interval required in this package insert.

Quality Control

- When new lot used, check or edit the quality control information.
- Scan the control barcode, choose corresponding quality control information and execute testing. For specific information on ordering quality controls, refer to the quality control section of the Analyzer Operating Instructions.

Sample Testing

- After successfully loading the sample, select the sample in interface and edit the assay for the sample to be tested and execute testing. For specific information on ordering patient specimens, refer to the sample ordering section of the Analyzer Operating Instructions.

To ensure proper test performance, strictly adhere to Analyzer Operating Instructions.

265 C-P-IFU-en-EU, V2.0, 2022-09

