



**[INTENDED USE]**

Central Nervous System Specific Protein S100β (CLIA) is a Chemiluminescence Immunoassay test for the quantitative measurement of Central Nervous System Specific Protein S100β (S100β) in human serum, which is intended to aid in the management of patients after potential brain injury.

The test must be performed on the ACCRE series analyzers (including ACCRE 6, ACCRE 8, ACCRE 90, ACCRE 100, ACCRE 120, and all other ACCRE series analyzers).

For *in vitro* diagnostic use only. For professional use only.

**[SUMMARY]**

S100 proteins are a family of small, dimeric multigenic calcium-binding proteins comprising various combinations of α and β subunits. S100 proteins most commonly occur as S100αα or S100αβ and S100ββ subtypes. Among them, S100αβ and S100ββ are commonly referred to as S100β protein. S100β is predominately expressed by cells of the central nervous system, mainly astroglial and Schwann cells, but is also expressed to some extent in other tissues, such as chondrocytes, melanocytes, muscle tissue and adipocytes. Severe brain injury invariably results in neuronal destruction and destabilization of the blood–brain barrier (BBB). These phenomena are accompanied by a release of S100β protein into the blood. S100β can be detected in patients with cerebral damage caused by several events, e.g. traumatic brain injuries(TBI) or stroke, and the extent of S100β elevation has been found to be useful in monitoring TBI progression.

**[ASSAY PRINCIPLES]**

Central Nervous System Specific Protein S100β (CLIA) hereinafter also referred to as the “S100β” is a one-step sandwich immunoassay to determine the level of Central Nervous System Specific Protein S100β.

Reagents for the assay are ready-to-use and pre-dispensed in the sealed reagent cartridge. All the assay steps are performed automatically by the ACCRE analyzer.

The sample is manually pipetted into the sample well of reagent cartridge, then sample, S100β antibody labeled with alkaline phosphatase and S100β antibody coated magnetic particles are mixed thoroughly together, incubating at 37°C. The analyte in the sample, labeled antibody and antibody coated with magnetic particles are bonded together as antibody-antigen sandwich immuno-complex. The immuno-complex is then washed in the magnetic field to eliminate the unbound components in the sample. The substrate is added to trigger the chemiluminescent reaction, the result of chemiluminescent reaction is measured by photomultiplier tube (PMT) as relative light units (RLUs). The concentration of the substance to be detected in the sample is proportional to the luminescence intensity. At the end, the results are automatically calculated by the ACCRE analyzers in relation to the calibration curve stored.

**[CONTENT OF THE KIT]**

**Package: 60 tests/kit with quality controls, 36 tests/kit with quality controls.**


**Traceability:**The calibrators of this kit can be traced to the internal reference substance. For specific calibrator concentration values, see the Calibrator Barcode Sheet for each lot.

| Contents                             | Quantities                   |                              | Notice                                                           |
|--------------------------------------|------------------------------|------------------------------|------------------------------------------------------------------|
| Brand: Tisenc                        | <b>REF</b><br>W61A           | <b>REF</b><br>W61C           |                                                                  |
| S100β reagent cartridge              | 60 strips                    | 36 strips                    | Ready to use                                                     |
| S100β Calibrator 1 (CAL 1)           | Dissolved to 1.0 mL x 1 vial | Dissolved to 1.0 mL x 1 vial | Lyophilized powder, Tris buffer, ProClin300, BSA, S100β antigen. |
| S100β Calibrator 2 (CAL 2)           | Dissolved to 1.0 mL x 1 vial | Dissolved to 1.0 mL x 1 vial | Lyophilized powder, Tris buffer, ProClin300, BSA, S100β antigen. |
| S100β Quality Control level 1 (QC 1) | Dissolved to 1.0 mL x 1 vial | Dissolved to 1.0 mL x 1 vial | Lyophilized powder, Tris buffer, ProClin300, BSA, S100β antigen. |
| S100β Quality Control level 2 (QC 2) | Dissolved to 1.0 mL x 1 vial | Dissolved to 1.0 mL x 1 vial | Lyophilized powder, Tris buffer, ProClin300, BSA, S100β antigen. |
| Instruction for Use                  | 1                            | 1                            | N/A                                                              |
| Reagent Registration Card            | 1                            | 1                            | N/A                                                              |
| Calibrator Barcode Sheet             | 1                            | 1                            | N/A                                                              |
| Quality Control Sheet                | 1                            | 1                            | N/A                                                              |

**Note: Refer to the target value list of the controls from Quality Control Sheet within the kit.**

**The Reagent Cartridge**

The strip consists of 19 wells. The 1st well is the sample well and the last 19th well is the reading well for chemiluminescent signal. From the 2nd to the 18th well are all covered with a labeled, aluminum foil seal. The label comprises a barcode which mainly indicates the reagent assay information, reagent lot number and expiration date. All reagents components needed for the assay are ready-to-use and pre-dispensed in the cartridges. And please do not interchange integral component from different reagents or lots.

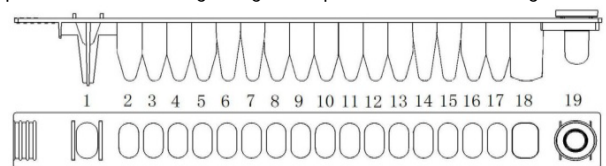

**Description of the reagent cartridge**

| Well           | Component          | Content                                                                                 | Volume |
|----------------|--------------------|-----------------------------------------------------------------------------------------|--------|
| 5              | Enzyme Label       | ALP (alkaline phosphatase) labeled anti-S100β antibody, MES Buffer, ProClin300.         | 50μL   |
| 7              | Magnetic Particles | Anti-S100β antibody coated magnetic particles, MOPS buffer, ProClin300.                 | 50μL   |
| 10, 11, 12, 14 | Wash Solutions     | Tris buffer, ProClin300, BND.                                                           | 350μL  |
| 16             | Substrate Solution | Disodium [(4-chlorophenyl)sulfanyl] (10-methyl-9(10H)-acridinylidene) methyl phosphate. | 200μL  |

Note: Well 1,2,3,4,6,8,9,13,15,17,18 are empty.

**[MATERIALS AND DISPOSABLES REQUIRED BUT NOT PROVIDED]**

- Pipette with disposable tip to dispense 200 μL.
- Powderless and disposable gloves.
- Disposable tips for ACCRE analyzer.
- For other specific materials and disposables, please refer to the Instrument User Manual.
- Instruments of the ACCRE series analyzers.

**[WARNINGS AND PRECAUTIONS]**

- For *in vitro* diagnostic use only.
- For professional use only by qualified laboratory personnel.
- This kit contains products of animal origin. Certified knowledge of the origin and/or sanitary state of the animals does not totally guarantee the absence of transmissible pathogenic agents. It is therefore recommended that these products be treated as potentially infectious, and handled observing the usual safety precautions (do not ingest; do not inhale).
- The kit contains products of human origin. No known analysis method can totally guarantee the absence of transmissible pathogenic agents. It is therefore recommended that these products be treated as potentially infectious and handled observing the usual safety precautions (see Laboratory Biosafety Manual - WHO -Geneva - latest edition)• Do not use reagents after the expiration date indicated on the box label.
- Do not use if the packaging of reagent cartridge is damaged or leaked.
- Do not use the reagents if the quantity is obviously insufficient.
- If any component adheres to inside wall of wells or aluminum film of the reagent cartridge, please gently switch the reagent cartridge to let the component back the wells.
- Do not reverse the reagent cartridge. Otherwise, the result will be unreliable.
- Package insert instructions must be carefully followed. Reliability of assay results cannot be guaranteed if there are any deviations from the instructions in this package insert.
- Use powderless gloves, as powder has been reported to cause false results for certain enzyme immunoassay tests.

**[STORAGE]**

- Store the Central Nervous System Specific Protein S100β (CLIA) upright at 2-8 °C, kit can be stored for 18 months from manufactured date . After the reagent cartridge is taken out from the refrigerator to equilibrate, the testing should be completed within 2 hours.
- Store the calibrators and quality controls at 2-8 °C till expiry date. When the calibrators and/or controls opened and re-dissolved, the testing should be completed within 2 hours, and the calibrators and/or controls will be stable

for 30 days at -20℃±5℃, without exceeding 1 freeze/thaw cycles.

- Keep away from sunlight.
- Do not freeze reagents cartridge.

**[SAMPLES]**

The sample volume for Central Nervous System Specific Protein S100β (CLIA) assay is 30μL, please note that at least 55μL of sample has to be pipetted manually into the reagent cartridge.

**Sample type and collection**

Human serum.  
Types of tubes:  
- Standard sampling tube;  
- Plastic tube with clot activator,  
- Plastic tube with clot activator and separation gel,  
It is recommended to validate collection tubes before using them as some may contain substances which interfere with test results.  
Note: Blood sampling tube results may vary from one manufacturer to another depending on the materials and additives used.  
It is the responsibility of each laboratory to validate the type of sample tube used and to follow the manufacturer's recommendations for use.

**Sample preparation**

The current revision of the WHO/DIL/LAB/99.1 document provides recommendations for the preparation of samples.  
For use of the sample tubes, follow the tube manufacturer's instructions for use.  
The pre-analytical step, including the preparation of blood samples, is an essential part of medical analyses. In accordance with Good Laboratory Practice, this step is performed under the responsibility of the laboratory manager.  
Samples containing suspended fibrin particles or erythrocyte stroma should be centrifuged before testing.  
If there is a lipid layer over the sample after centrifuging, please transfer the clear sample into a new sample tube without transferring the lipid layer.  
Severe hemolysis samples or heat-inactivated samples are not recommended for applying on the reagents.  
Preparation of frozen-stored samples: after thawing, these samples must be thoroughly mixed before testing.  
Mix using a vortex-type mixer. If necessary, clarify samples by centrifuging before testing.

**Sample stability**

Samples (serum) can be stored at room temperature (15-30 °C) for up to 7 hours or at 2-8 °C for up to 24 hours; if longer storage is required, freeze the serum at -20 °C or below -20 °C for up to 3 months, without exceeding 1 freeze/thaw cycle.

**[INSTRUCTIONS FOR USE]**

For complete instructions, see the Instrument User Manual.  
**When using the assay for the first time.** Register the assay using the registration code in the reagent kit before any assay is used in the ACCRE analyzer for the first time. Calibration, QC and sample test are only available after assay was registered successfully.  
**When opening a new lot of reagents.** Each new lot of reagent must be registered by using the registration code in Reagent Registration Card within the reagent kit.

**Calibration**

Calibration must be performed with the reagents and calibrators from Manufacturer.

Each bottle of lyophilized calibrator should be re-dissolved by distilled water, deionized water, or purified water to 1.0mL before use. Mix it thoroughly and avoid foam formation. Keep dissolved calibrators upright for about 15 minutes to equilibrate to room temperature (15-30°C).

Calibration is necessary under the following conditions:

- 1). New reagent lot number applied;
- 2). **Same lot of reagents used more than 4 weeks;**
- 3). QC out of control;
- 4). When necessary, for example: system maintenance or repair that may affect the analytic performance.

#### Test Procedure

1. Remove the kit from storage at 2-8 °C and take out the required number of reagent cartridges. Allow the reagent cartridges, specimens, calibrators, and/or controls to equilibrate to room temperature (15-30°C) prior to testing.
2. Use each assay strips for each sample, calibrator or control respectively.
3. Place the reagent cartridges into the reagent rack.
4. Manually pipette the Sample, Calibrator, or Control into the sample well of each reagent cartridge .

**Please note that at least 55µL sample has to be pipetted manually into the reagent cartridge and 30µL sample is needed for each test.**

**\* Before pipetting: Mix the samples using a vortex-type mixer if necessary, and ensure that the samples and sample diluent (if applicable) DO NOT have any bubbles. The same for calibrators and controls.**

5. Put the sample rack into the reagent chamber of ACCRE analyzer.
6. Place the assay tips into the reagent chamber of ACCRE analyzer.
7. Input Sample ID, sample type, dilution etc. into the software as directed by USER MANUAL.
8. Central Nervous System Specific Protein S100β (CLIA) assay will be performed automatically by the ACCRE analyzer.
9. After assay is completed, remove the reagent rack from the ACCRE analyzer. Dispose the used reagent cartridge into an appropriate recipient.

#### Quality control

Each bottle of lyophilized Quality Control should be re-dissolved by water, deionized water, or purified water to 1.0mL before use. Mix it thoroughly and avoid foam formation. Keep dissolved quality controls upright for about 15 minutes to equilibrate to room temperature (15-30°C).

Quality control should be performed in accordance with local regulations or requirements related to accreditation, as well as requirements defined in the laboratory's quality control procedure.

#### [RESULTS AND INTERPRETATION]

##### Calculation of Results

The results are automatically calculated by the ACCRE analyzer using stored calibration curve which generate by 2-point calibration, and calculated according to a predefined mathematical model, the results expressed in µg/L.

##### Interpretation of Results

The detection range is from 0.020 µg/L to 39.000 µg/L. For samples with S100β below the upper limit of detection, quantitative determination can be performed; if the sample is lower than the lower limit of detection, the reported result is <0.020 µg/L; if the sample is higher than the upper limit of detection, the reported result is >39.000 µg/L.

Results of Central Nervous System Specific Protein S100β (CLIA) assays were determined by testing serum specimens from a study in clinical centers with group of totally 278 healthy individuals in China. The 95<sup>th</sup> percentile value

is 0.110 µg/L.

**\* Please note that results may differ between laboratories due to variations in population and test method. If necessary, each laboratory should establish its own reference range.**

In interpreting the results, patient's clinical condition including symptoms, diseases history, and other relevant data and information should be considered.

#### [LIMITATIONS]

- 1.For diagnostic purposes, results should be used in conjunction with other data; e.g., symptoms, results of other tests, clinical impressions, etc.
2. Please note that heterophile antibody in human serum can interfere the test result.
3. High concentration of HAMA can also cause a false positive or false negative result.
4. If the test results are inconsistent with clinical evidence, additional testing is suggested to confirm the result.

#### [PERFORMANCE CHARACTERISTICS]

##### Detection Limit

Limit of Blank (LoB): 0.020 µg/L

The Limit of Blank was determined in accordance with the CLSI® EP17-A2 recommendations.

##### Measurement range

The measurement range is the range of values corresponding to the acceptable performance limits (precision and linearity).

**The measurement range of the Central Nervous System Specific Protein S100β (CLIA) assay is 0.020 µg/L~39.000 µg/L.**

##### Linearity

Linearity was evaluated according to CLSI® EP06-A recommendations. The Central Nervous System Specific Protein S100β (CLIA) assay is between 0.020 µg/L~39.000 µg/L. The absolute linear correlation coefficient (r) should be bigger than 0.9900.

##### Precision

The precision study was performed according to CLSI® EP05-A3 recommendations. The within-lot precision CV% is less than 10%.

##### Interfering Substances

Potential interference substances were studied according to CLSI ® EP07-A3 recommendations. No significant interference was detected up to the maximum concentrations tested.

| Substance         | Concentration |
|-------------------|---------------|
| Hemoglobin        | ≤ 200 mg/dL   |
| Bilirubin         | ≤ 10 mg/dL    |
| Triglyceride      | ≤ 1000 mg/dL  |
| Rheumatoid factor | ≤ 100 IU/mL   |

##### Hook effect

No hook effect was found up to S100β concentrations of 1000 µg/L.

##### Accuracy

The relative deviation between the measured value and the calibration concentration is within the range of ±15%.

#### HAMA

Patient samples containing human anti-mouse antibodies (HAMA) may give falsely elevated or decreased values. Although HAMA-neutralizing agents are added, extremely high HAMA serum concentrations may occasionally influence results.

#### [WASTE DISPOSAL]

Dispose used or unused reagents as well as any other contaminated disposable materials following procedures for infectious or potentially infectious products.

It is the responsibility of each laboratory to handle waste and effluents produced according to their nature and degree of hazardousness and to treat and dispose of them (or have them treated and disposed of) in accordance with any applicable regulations.

#### [INDEX OF SYMBOLS]

| Symbol                                                                              | Meaning                                                             | Symbol                                                                              | Meaning                                                           | Symbol                                                                              | Meaning                                       |
|-------------------------------------------------------------------------------------|---------------------------------------------------------------------|-------------------------------------------------------------------------------------|-------------------------------------------------------------------|-------------------------------------------------------------------------------------|-----------------------------------------------|
|  | Consult instructions for use                                        |  | Contains sufficient for <n> tests                                 |  | Date of manufacture                           |
|  | <i>In vitro</i> diagnostic medical device                           |  | Use-by date                                                       |  | Do not re-use                                 |
|  | Temperature limit                                                   |  | Batch code                                                        |  | Catalogue number                              |
|  | Keep away from sunlight                                             |  | Keep dry                                                          |  | Manufacturer                                  |
|  | Authorized representative in the European Community/ European Union |  | Do not use if package is damaged and consult instructions for use |  | Contains biological material of animal origin |
|  | Biological risks                                                    |  | Caution                                                           |  | CE marking                                    |
|  | Upright                                                             |  | Fragile, handle with care                                         |  | Do not roll                                   |
|  | Control                                                             |                                                                                     |                                                                   |                                                                                     |                                               |

#### [REFERENCES]

1. Moore BW. A sluble protein characteristic of the nervous system. Biochem Biophys Res Comm 1965;19:739-744.
2. Bloomfield S M, Mckinney J, Smith L ,et al.Reliability of S100B in predicting severity of central nervous system injury[J].Neurocritical Care, 2007, 6(2):121-138.
3. Murillo-Cabezas F , Muñoz-Sánchez, Maria ángeles,Rincón-Ferrari, Maria Dolores,et al.The prognostic value of the temporal course of S100beta protein in post-acute severe brain injury: A prospective and observational study.[J].Brain injury : BI, 2010, 24(4):609-19.
4. K. Nylén, St M , Csajbok L Z ,et al.Serum levels of S100B, S100A1B and S100BB are all related to outcome after severe traumatic brain injury[J].Acta Neurochirurgica, 2008, 150(3):p.221-227.
5. Hayashi K , Hoshida Y , Horie Y ,et al.Immunohistochemical study on the distribution of α and β subunits of S-100 protein in brain tumors[J].Acta Neuropathologica, 2004, 81:657-663.
6. Abraha H D , Butterworth R J , Bath P M W ,et al.Serum S-100 Protein, Relationship to Clinical Outcome in Acute Stroke[J].Annals of Clinical Biochemistry, 1997, 34(4):366-370.
7. Haimoto H , Hosoda S , Kato K. Differential distribution of immunoreactive

S100-alpha and S100-beta proteins in normal nonnervous human tissues.[J].Laboratory investigation; a journal of technical methods and pathology, 1987, 57(5):489.

8. Boscato Lm,Stuart MC.Heterophilic antibodies: a problem for all immunoassays. Clin Chem 1988;34:27-33.

9. Kricka L.Interferences in immunoassays - still a threat. Clin Chem 2000; 46:1037-1038.

10. Bjerner J, et al. Immunometric assay interference: incidence and prevention. Clin Chem 2002;48:613-621.

 Shenzhen Tisenc Medical Devices Co., Ltd.  
Address: 11F, 11G, Kechuang Building, Quanzhi Science and Innovation Park, Maozhoushan Industrial Park, Shajing Community, BaoAn District, Shenzhen, 518104, P.R.China  
Phone: +86-400-830-8768 +86-0755-23225549  
E-mail: sales@tisenc.com

**EC REP**

CMC Medical Devices & Drugs S.L.  
C/ Horacio Lengo no 18, CP 29006, Málaga, Spain

Eng 20220330 -Ver 1.0