



MAGLUMI® PTH (CLIA)

INTENDED USE

The kit is an *in vitro* chemiluminescence immunoassay for the quantitative determination of bioactive parathyroid hormone PTH(1-84) in human serum and plasma using the MAGLUMI series Fully-auto chemiluminescence immunoassay analyzer and Biolumi series Integrated System, and the assay is used for an aid in the diagnosis of individuals with suspected or confirmed parathyroid disease.

SUMMARY

Parathyroid hormone (PTH) 1-84 is an 84-amino acid single-chain polypeptide that is synthesized and secreted by chief cells of the parathyroid glands, whose metabolism is regulated by blood ionized calcium concentration^{1,2}. The principal effect of PTH is to increase the extracellular fluid (ECF) calcium concentration by mobilizing calcium from bone, increasing tubular calcium reabsorption, and indirectly on the gut, by increasing calcitriol synthesis, which in turn increases intestinal absorption of calcium and exerts an endocrine feed-back on the secretion of PTH at the parathyroid level^{1,3}. PTH also decreases the renal reabsorption of phosphate in the proximal tubule, thereby decreasing serum phosphate³. The half-life of the whole molecule of PTH(1-84) is between 2 and 5 minutes, once the secreted 1-84 PTH reaches the circulation, it is rapidly taken up by the liver and the kidney and further metabolized into different fragments, including PTH(7-84)⁴⁻⁵.

Hyperparathyroidism or hypoparathyroidism can cause abnormally elevated or inadequate serum parathyroid hormone levels, leading to hypercalcemia or hypocalcemia⁶⁻⁹. Intraoperative parathyroid hormone (IO-PTH) measurements have been proposed to improve operative success rates in primary, secondary and tertiary hyperparathyroidism (PHP, SHP and THP)⁶, and avoid early termination of surgery as well as necessity for reoperation¹⁰. PTH measurement is routinely prescribed in patients with chronic renal failure (CRF) to identify renal osteodystrophy (ROD) subtypes and adapt treatment, and in non-renal patients to explore any disorder of calcium-phosphate metabolism³. In patients with chronic kidney disease (CKD), PTH measurements are widely used as a surrogate marker in the assessment of the skeletal and mineral disorders associated with CKD or CKD-mineral and bone disorders (CKD-MBD)¹¹. The MAGLUMI PTH assay is a third generation PTH assay as it specifically measures the biologically intact molecule of PTH, PTH(1-84)³. This can be of advantage in chronic renal failure patients as it was shown that small and large C-PTH fragments accumulate in advanced renal failure¹². There is evidence that 1-84 PTH decreases more rapidly than "non-1-84" PTH immediately after parathyroidectomy³, and the PTH(1-84)/intact-PTH ratio correlates with the severity of hyperparathyroidism in haemodialysis (HD) patients¹³.

Therefore, accurate detection of blood PTH levels is crucial for evaluating parathyroid function and for the clinical management of patients with CKD-MBD⁵.

TEST PRINCIPLE

Sandwich chemiluminescence immunoassay.

The sample, buffer, magnetic microbeads coated with PTH(1-84) monoclonal antibody are mixed thoroughly, incubating and performing a wash cycle after a precipitation in a magnetic field. ABEI labeled with another PTH(1-84) monoclonal antibody are then added, reacting to form sandwich complexes and incubating. After precipitation in a magnetic field, the supernatant is decanted and then a wash cycle is performed. Subsequently, the Starter 1+2 are added to initiate a chemiluminescent reaction. The light signal is measured by a photomultiplier as relative light units (RLUs), which is proportional to the concentration of PTH(1-84) present in the sample.

REAGENTS

Kit Contents

Component	Description	100 tests/kit	50 tests/kit
Magnetic Microbeads	Magnetic microbeads coated with PTH(1-84) monoclonal antibody (~12.0 µg/mL) in PBS buffer, NaNa ₃ (<0.1%).	2.5 mL	1.5 mL
Calibrator Low	A low concentration of PTH(1-84) antigen in PBS buffer, NaNa ₃ (<0.1%).	1.5 mL	1.5 mL
Calibrator High	A high concentration of PTH(1-84) antigen in PBS buffer, NaNa ₃ (<0.1%).	1.5 mL	1.5 mL
Buffer	PBS buffer, NaNa ₃ (<0.1%).	6.5 mL	4.0 mL
ABEI Label	ABEI labeled with PTH(1-84) monoclonal antibody (~0.400 µg/mL) in PBS buffer, NaNa ₃ (<0.1%).	7.5 mL	4.5 mL
Control 1	A low concentration of PTH(1-84) antigen (45.0 pg/mL) in PBS buffer, NaNa ₃ (<0.1%).	1.5 mL	1.5 mL
Control 2	A high concentration of PTH(1-84) antigen (140 pg/mL) in PBS buffer, NaNa ₃ (<0.1%).	1.5 mL	1.5 mL
All reagents are provided ready-to-use.			

Warnings and Precautions

- For *in vitro* diagnostic use.
 - For professional use only.
 - Exercise the normal precautions required for handling all laboratory reagents.
 - Personal protective measures should be taken to prevent any part of the human body from contacting samples, reagents, and controls, and should comply with local operating requirements for the assay.
 - A skillful technique and strict adherence to the package insert are necessary to obtain reliable results.
 - Do not use kit beyond the expiration date indicated on the label.
 - Do not interchange reagent components from different reagents or lots.
 - Avoid foam formation in all reagents and sample types (specimens, calibrators and controls).
 - All waste associated with biological samples, biological reagents and disposable materials used for the assay should be considered potentially infectious and should be disposed of in accordance with local guidelines.
 - This product contains sodium azide. Sodium azide may react with lead or copper plumbing to form highly explosive metal azides. Immediately after disposal, flush with a large volume of water to prevent azide build-up. For additional information, see Safety Data Sheets available for professional user on request.
- Note: If any serious incident has occurred in relation to the device, please report to Shenzhen New Industries Biomedical Engineering Co., Ltd. (Snibe) or our authorized representative and the competent authority of the Member State in which you are established.

Reagent Handling

- To avoid contamination, wear clean gloves when operating with a reagent kit and sample. When handling reagent kit, replace the gloves that have been in contact with samples, since introduction of samples will result in unreliable results.
- Do not use kit in malfunction conditions; e.g., the kit leaking at the sealing film or elsewhere, obviously turbid or precipitation is found in reagents (except for Magnetic Microbeads) or control value is out of the specified range repeatedly. When kit in malfunction conditions, please contact Snibe or our authorized distributor.
- To avoid evaporation of the liquid in the opened reagent kits in refrigerator, it is recommended that the opened reagent kits to be sealed with reagent seals contained within the packaging. The reagent seals are single use, and if more seals are needed, please contact Snibe or our authorized distributor.
- Over time, residual liquids may dry on the septum surface. These are typically dried salts and have no effect on assay efficacy.
- Use always the same analyzer for an opened reagent integral.
- For magnetic microbeads mixing instructions, refer to the Preparation of the Reagent section of this package insert.
- For further information about the reagent handling during system operation, please refer to Analyzer Operating Instructions.

Storage and Stability

- Do not freeze the integral reagents.
- Store the reagent kit upright to ensure complete availability of the magnetic microbeads.



0123



130211008M:100 tests/kit
130611008M: 50 tests/kit

- Protect from direct sunlight.

Stability of the Reagents	
Unopened at 2-8°C	until the stated expiration date
Opened at 2-8°C	6 weeks
On-board	4 weeks

Stability of Controls	
Unopened at 2-8°C	until the stated expiration date
Opened at 10-30°C	6 hours
Opened at 2-8°C	6 weeks
Frozen at -20°C	3 months
Frozen and thawed cycles	no more than 3 times

SPECIMEN COLLECTION AND PREPARATION

Specimen Types

Only the specimens listed below were tested and found acceptable.

Specimen Types	Collection Tubes
Serum	Tubes without additive/accessory, or tubes containing clot activator or clot activator with gel
Plasma	K2-EDTA, K3-EDTA or Li-heparin

- The sample types listed were tested with a selection of sample collection tubes that were commercially available at the time of testing, i.e. not all available tubes of all manufacturers were tested. Sample collection systems from various manufacturers may contain differing materials which could affect the test results in some cases. Follow tube manufacturers' instructions carefully when using collection tubes.

Specimen Conditions

- Do not use heat-inactivated samples or grossly hemolyzed/hyperlipidaemia specimens and specimens with obvious microbial contamination.
- Ensure that complete clot formation in serum specimens has taken place prior to centrifugation. Some serum specimens, especially those from patients receiving anticoagulant or thrombolytic therapy, may exhibit increased clotting time. If the serum specimen is centrifuged before a complete clotting, the presence of fibrin may cause erroneous results.
- Samples must be free of fibrin and other particulate matter.
- To prevent cross contamination, use of disposable pipettes or pipette tips are recommended.

Preparation for Analysis

- Inspect all specimens for foam. Remove foam with an applicator stick before analysis. Use a new applicator stick for each specimen to prevent cross contamination.
- Frozen specimens must be completely thawed before mixing. Mix thawed specimens thoroughly by low speed vortexing or by gently inverting. Visually inspect the specimens. If layering or stratification is observed, mix until specimens are visibly homogeneous. If specimens are not mixed thoroughly, inconsistent results may be obtained.
- For reliable results, specimens should be free of fibrin, red blood cells, or other particulate matter. Specimens must be centrifuged prior to testing, transfer clarified specimen to a sample cup or secondary tube for testing. For centrifuged specimens with a lipid layer, transfer only the clarified specimen and not the lipemic material.
- The sample volume required for a single determination of this assay is 100 µL.

Specimen Storage

Serum specimens removed from the separator, red blood cells or clot may be stored up to 7 hours at 10-30°C or 24 hours at 2-8°C, or 6 months frozen at -20°C. Frozen specimens subjected to 1 freeze/thaw cycles have been evaluated. Plasma specimens removed from the separator, red blood cells or clot may be stored up to 24 hours at 10-30°C or 48 hours at 2-8°C, or 6 months frozen at -20°C. Frozen specimens subjected to 1 freeze/thaw cycles have been evaluated.

Specimen Shipping

- Package and label specimens in compliance with applicable local regulations covering the transport of clinical specimens and infectious substances.
- Do not exceed the storage limitations listed above.

Specimen Dilution

- Samples, with PTH(1-84) concentrations above the analytical measuring interval, can be diluted with Diluent by manual dilution procedure. The recommended dilution ratio is 1:2(sample volume:total volume). The concentration of the diluted sample must be >1150 pg/mL.
- For manual dilution, multiply the result by the dilution factor. Please choose applicable diluents or ask Snibe for advice before manual dilution.

PROCEDURE

Materials Provided

PTH (CLIA) assay, control barcode labels.

Materials Required (But Not Provided)

- General laboratory equipment.
- Fully-auto chemiluminescence immunoassay analyzer Maglumi 600, Maglumi 800, Maglumi 1000, Maglumi 2000, Maglumi 2000 Plus, Maglumi 4000, Maglumi 4000 Plus, MAGLUMI X3, MAGLUMI X6, MAGLUMI X8, MAGLUMI X10, or Integrated System Biolumi 8000 and Biolumi CX8.
- Additional accessories of test required for the above analyzers include Reaction Module, Starter 1+2, Wash Concentrate, Light Check, Tip, and Reaction Cup. Specific accessories and accessories' specification for each model refer to corresponding Analyzer Operating Instructions.
- Please use accessories specified by Snibe to ensure the reliability of the test results.

Assay Procedure

Preparation of the Reagent

- Take the reagent kit out of the box and visually inspect the integral vials for leaking at the sealing film or elsewhere. If there is no leakage, please tear off the sealing film carefully.
- Open the reagent area door; hold the reagent handle to get the RFID label close to the RFID reader (for about 2s); the buzzer will beep; one beep sound indicates successful sensing.
- Keeping the reagent straight insert to the bottom along the blank reagent track.
- Observe whether the reagent information is displayed successfully in the software interface, otherwise repeat the above two steps.
- Resuspension of the magnetic microbeads takes place automatically when the kit is loaded successfully, ensuring the magnetic microbeads are totally resuspended homogenous prior to use.

Assay Calibration

- Select the assay to be calibrated and execute calibration operation in reagent area interface. For specific information on registering calibrations, refer to the calibration section of Analyzer Operating Instructions.
- Execute recalibration according to the calibration interval required in this package insert.

Quality Control

- When new lot used, check or edit the quality control information.
- Scan the control barcode, choose corresponding quality control information and execute testing. For specific information on registering quality controls, refer to the quality control section of the Analyzer Operating Instructions.

Sample Testing

After successfully loading the sample, select the sample in interface and edit the assay for the sample to be tested and execute testing. For specific information on registering samples, refer to the sample section of the Analyzer Operating Instructions.

To ensure proper test performance, strictly adhere to Analyzer Operating Instructions.

1031 PTH(1-84)-IFU-en-EU, V3.0, 2025-04

Calibration

Traceability: This method has been standardized against the WHO International Standard 95/646.
Test of assay specific calibrators allows the detected relative light unit (RLU) values to adjust the master curve.
Recalibration is recommended as follows:
• Whenever a new lot of Reagent or Starter 1+2 is used.
• Every 28 days.
• The analyzer has been serviced.
• Control values lie outside the specified range.

Quality Control

Controls are recommended for the determination of quality control requirements for this assay and should be run in singlicate to monitor the assay performance. Refer to published guidelines for general quality control recommendations, for example Clinical and Laboratory Standards Institute (CLSI) Guideline C24 or other published guidelines¹⁴.

Quality control is recommended once per day of use, or in accordance with local regulations or accreditation requirements and your laboratory's quality control procedures, quality control could be performed by running the PTH(1-84) assay:

- Whenever the kit is calibrated.
 - Whenever a new lot of Starter 1+2 or Wash Concentrate is used.
- Controls are only applicable with MAGLUMI and Biolumi systems and only used matching with the same top eight LOT numbers of corresponding reagents. For each target value and range refer to the label.

The performance of other controls should be evaluated for compatibility with this assay before they are used. Appropriate value ranges should be established for all quality control materials used.

Control values must lie within the specified range, whenever one of the controls lies outside the specified range, calibration should be repeated and controls retested. If control values lie repeatedly outside the predefined ranges after successful calibration, patient results must not be reported and take the following actions:

- Verify that the materials are not expired.
- Verify that required maintenance was performed.
- Verify that the assay was performed according to the package insert.
- If necessary, contact Snibe or our authorized distributors for assistance.

If the controls in kit are not enough for use, please order PTH (CLIA) Controls (REF: 1602011031MT) from Snibe or our authorized distributors for more.

RESULTS

Calculation

The analyzer automatically calculates the PTH(1-84) concentration in each sample by means of a calibration curve which is generated by a 2-point calibration master curve procedure. The results are expressed in pg/mL. If pmol/L is used to report results, manual conversion is required, and the conversion formula is:

pg/mLx0.106=pmol/L pmol/Lx9.43=pg/mL

For further information please refer to the Analyzer Operating Instructions.

Interpretation of Results

The expected range for the PTH(1-84) assay was obtained by testing 300 apparently healthy individuals, gave the following expected value: 15-57 pg/mL (95th percentile).

Results may differ between laboratories due to variations in population and test method. It is recommended that each laboratory establish its own reference interval.

LIMITATIONS

- Results should be used in conjunction with patient's medical history, clinical examination and other findings.
- If the PTH(1-84) results are inconsistent with clinical evidence, additional testing is needed to confirm the result.
- Specimens from patients who have received preparations of mouse monoclonal antibodies for diagnosis or therapy may contain human anti-mouse antibodies (HAMA). Such specimens may show either falsely elevated or depressed values when tested with assay kits which employ mouse monoclonal antibodies^{15,16}. Additional information may be required for diagnosis.
- Heterophilic antibodies in human serum can react with reagent immunoglobulins, interfering with *in vitro* immunoassays. Patients routinely exposed to animals or animal serum products can be prone to this interference and anomalous values may be observed¹⁷.
- Bacterial contamination or heat inactivation of the specimens may affect the test results.

SPECIFIC PERFORMANCE CHARACTERISTICS

Representative performance data are provided in this section. Results obtained in individual laboratories may vary.

Precision

Precision was determined using the assay, samples and controls in a protocol (EP05-A3) of the CLSI (Clinical and Laboratory Standards Institute): duplicates at two independent runs per day for 5 days at three different sites using three lots of reagent kits (n = 180). The following results were obtained:

Sample	Mean (pg/mL) (n=180)	Within-Run		Between-Run		Reproducibility	
		SD (pg/mL)	%CV	SD (pg/mL)	%CV	SD (pg/mL)	%CV
Serum Pool 1	19.181	0.919	4.79	0.452	2.36	1.714	8.94
Serum Pool 2	56.046	1.610	2.87	1.753	3.13	4.257	7.60
Serum Pool 3	594.995	20.170	3.39	20.042	3.37	43.476	7.31
Plasma Pool 1	19.473	0.967	4.97	0.399	2.05	1.785	9.17
Plasma Pool 2	62.371	1.702	2.73	1.443	2.31	4.790	7.68
Plasma Pool 3	588.436	20.852	3.54	20.498	3.48	41.415	7.04
Control 1	46.126	2.357	5.11	0.701	1.52	3.769	8.17
Control 2	135.109	4.844	3.59	5.026	3.72	10.779	7.98

Based on internal testing on the MAGLUMI and Biolumi systems, the overall repeatability (% CV) is estimated to be ≤ 5% for samples tested, the overall reproducibility (%CV) is estimated to be ≤ 10%. Performance of the assay at individual laboratories may vary.

Linear Range

10.0-2300 pg/mL (defined by the Limit of Quantitation and the maximum of the master curve).

Reportable Interval

5.50-4600 pg/mL (defined by the Limit of Detection and the maximum of the master curve×Recommended Dilution Ratio).

Analytical Sensitivity

Limit of Blank (LoB) ≈2.50 pg/mL.
Limit of Detection (LoD) ≈5.50 pg/mL.
Limit of Quantitation (LoQ) ≈10.0 pg/mL.

Analytical Specificity

Interference

Interference was determined using the assay, three samples containing different concentrations of analyte were spiked with potential endogenous and exogenous interferences in a protocol (EP7-A2) of the CLSI. The measurement deviation of the interference substance is within ±10%. The following results were obtained:

Interference	No interference up to
Bilirubin	65 mg/dL
Hemoglobin	875 mg/dL
Intralipid	1400 mg/dL
HAMA	40 ng/mL
Rheumatoid factor	1500 IU/mL

ANA	398 AU/mL
Biotin	50 µg/mL

Cross-Reactivity

Cross-reactivity was determined using the assay, three samples containing different concentrations of analyte were spiked with potential cross-reactant in a protocol (EP7-A2) of the CLSI. The measurement deviation of the interference substance is within ±10%. The following results were obtained:

Cross-reactant	No interference up to
Risedronate	150 mg/L
Alendronate	350 mg/L
β-Estradiol	250 mg/L
β-Estradiol-17-valerate	250 mg/L
β-Estradiol 3-sulfate	250 mg/L
Osteocalcin	1000 ng/mL
β-CTx	10 ng/mL
P1NP	2000 ng/mL
PTH (1-34)	10000 pg/mL
PTH (39-84)	10000 pg/mL
PTH (7-84)	10000 pg/mL
PTH-RP	5000 pg/mL

High-Dose Hook

No high-dose hook effect was seen for PTH(1-84) concentrations up to 30000 pg/mL.

Method Comparison

A comparison of the PTH(1-84) assay with a commercially available immunoassay, gave the following correlations (pg/mL):

Number of samples measured: 118

Passing-Bablok: y=0.9927x+0.1486, r=0.962.

The clinical specimen concentrations were between 10.02 and 2297 pg/mL.

REFERENCES

1. Schenck P, Chew D, Nagode L, et al. Disorders of Calcium: Hypercalcemia and Hypocalcemia[J]. Fluid, electrolyte, and acid-base disorders in small animal practice, 2006, 4: 120-94..
2. Friedman P A, Goodman W G. PTH (1-84)/PTH (7-84): a balance of power [J]. American Journal of Physiology-Renal Physiology, 2006, 290(5):F975-84.
3. Souberbielle J C, Friedlander G, Cormier C. Practical considerations in PTH testing [J]. Clinica Chimica Acta, 2006, 366(1-2):81-89.
4. Torres P U. The need for reliable serum parathyroid hormone measurements[J]. Kidney international, 2006, 70(2): 240-243.
5. Huimin C , Ying C , Changying X , et al. Effects of Parathyroidectomy on Plasma iPTH and (1–84) PTH Levels in Patients with Stage 5 Chronic Kidney Disease[J]. Hormone and Metabolic Research, 2018, 50(10): 761-767.
6. Ohe M N, Santos R O, Kunii I S, et al. Usefulness of a rapid immunometric assay for intraoperative parathyroid hormone measurements[J]. Brazilian Journal of Medical and Biological Research, 2003, 36: 715-721.
7. Bilezikian J P, Khan A, Potts Jr J T, et al. Hypoparathyroidism in the adult: Epidemiology, diagnosis, pathophysiology, target - organ involvement, treatment, and challenges for future research[J]. Journal of Bone and Mineral Research, 2011, 26(10): 2317-2337.
8. Shoback D. Hypoparathyroidism [J]. New England Journal of Medicine, 2008, 359(4): 391–403.
9. Nussbaum SR, Potts JT. Immunoassays for parathyroid hormone 1-84 in the diagnosis of hyperparathyroidism [J]. Journal of Bone and Mineral Research, 1991, 6(S2):S43-S50.
10. Carter A B, Howanitz P J. Intraoperative testing for parathyroid hormone: a comprehensive review of the use of the assay and the relevant literature[J]. Archives of pathology & laboratory medicine, 2003, 127(11): 1424-1442.
11. Endres D B. Investigation of hypercalcemia[J]. Clinical biochemistry, 2012, 45(12): 954-963.
12. D'Amour P. Acute and chronic regulation of circulating PTH: Significance in health and in disease [J]. Clinical Biochemistry, 2012, 45(12):964-969.
13. Tanaka M, Komaba H, Itoh K, et al. The whole-PTH/intact-PTH ratio is a useful predictor of severity of secondary hyperparathyroidism[J]. NDT plus, 2008, 1(suppl. 3): iii59-iii62.
14. CLSI. Statistical Quality Control for Quantitative Measurement Procedures: Principles and Definitions. 4th ed. CLSI guideline C24. Wayne, PA: Clinical and Laboratory Standards Institute; 2016.
15. Robert W. Schroff, Kenneth A. Foon, Shannon M. Beatty, et al. Human Anti-Murine Immunoglobulin Responses in Patients Receiving Monoclonal Antibody Therapy[J]. Cancer Research, 1985, 45(2):879-885.
16. Primus F J, Kelley E A, Hansen H J, et al. "Sandwich"-type immunoassay of carcinoembryonic antigen in patients receiving murine monoclonal antibodies for diagnosis and therapy[J]. Clinical Chemistry, 1988, 34(2):261-264.
17. Boscatto L M , Stuart M C . Heterophilic antibodies: a problem for all immunoassays[J]. Clin Chem 1988;34(1):27-33.

SYMBOLS EXPLANATIONS

	Consult instructions for use		Manufacturer
	Temperature limit (Store at 2-8°C)		Use-by date
	Contains sufficient for<n> tests		Keep away from sunlight
	This way up		Authorized representative in the European Community
	<i>In vitro</i> diagnostic medical device		Kit component
	Catalogue number		Batch code
	CE marking with notified body ID number		

MAGLUMI® and Biolumi® are trademarks of Snibe. All other product names and trademarks are the property of their respective owners.

	Shenzhen New Industries Biomedical Engineering Co., Ltd. No.23, Jinxiu East Road, Pingshan District, 518122 Shenzhen, P.R. China Tel: +86-755-21536601 Fax:+86-755-28292740
	Shanghai International Holding Corp. GmbH (Europe) Eiffestrasse 80, 20537 Hamburg, Germany Tel: +49-40-2513175 Fax: +49-40-255726