



[INTENDED USE]

Thrombomodulin (CLIA) is a chemiluminescent immunoassay used to quantitatively determine thrombomodulin content in human plasma and whole blood, and is mainly used as an aid in the diagnosis of thrombotic diseases

The test must be performed on the ACCRE series of analyzers (including ACCRE 8, ACCRE 90, ACCRE 120, and all other ACCRE series analyzers).

For *in vitro* diagnostic use only. For professional use only.

[SUMMARY]

TM is a marker of vascular endothelial cell damage.

TM is a single-chain multi-segment transmembrane glycoprotein regulated by a series of mechanisms. About 99% of TM is expressed by vascular endothelial cells, but it is expressed in smooth muscle cells, platelets, monocytes, leukocytes, keratin cells, astrocytes and some tumor cells. It is an important intravascular clotting inhibitor that causes thrombin to change from procoagulant to anticoagulant. In the process of thrombosis formation, the vascular endothelial cells are destroyed, and TM is shed from the surface of the endothelial cells into the blood, resulting in increased plasma concentration. Therefore, the plasma TM value can be used as a sensitive indicator to evaluate the degree of vascular endothelial injury and the severity of the disease

[ASSAY PRINCIPLES]

Thrombomodulin (CLIA) hereinafter also referred to as the " TM " is a two-step sandwich method for the determination of thrombomodulin levels in human plasma and whole blood.

Reagents for the assay are ready-to-use and pre-dispensed in the sealed reagent cartridge. All the assay steps are performed automatically by the ACCRE analyzer.

The sample is manually pipetted into the sample well of reagent cartridge, then sample. In the first step, the sample to be tested and the sample treatment solution were added to the magnetic bead coated with TM antibody, which was mixed and incubated for cleaning; in the second step, the alkaline phosphatase-labeled TM antibody reaction was added, and the double antibody sandwich immune complex was formed after mixing and incubating. The immuno-complex is then washed in the magnetic field to eliminate the unbound components in the sample. The substrate is added to trigger the chemiluminescent reaction, the result of chemiluminescent reaction is measured by photomultiplier tube (PMT) as relative light units (RLUs). The concentration of the substance to be detected in the sample is proportional to the luminescence intensity. At the end, the results are automatically calculated by the ACCRE analyzers in relation to the calibration curve stored.

[CONTENT OF THE KIT]


Package: 60 tests/kit with quality controls, 36 tests/kit with quality controls. Traceability: The calibrators of this kit can be traced to the internal reference substance. For specific calibrator concentration values, see the Calibrator Barcode Sheet for each lot.

Contents	Quantities		Notice
	REF W160A	REF W160C	
TM reagent cartridge	60 strips	36 strips	Ready to use
TM Calibrator 1 (CAL 1)	Dissolved to 1.0 mL x 1 via	Dissolved to 1.0 mL x 1 via	Lyophilized powder, Tris buffer, ProClin300, TM antigen.
TM Calibrator 2 (CAL 2)	Dissolved to 1.0 mL x 1 via	Dissolved to 1.0 mL x 1 via	Lyophilized powder, Tris buffer, ProClin300, TM antigen.
TM Quality Control level 1 (QC 1)	Dissolved to 1.0 mL x 1 via	Dissolved to 1.0 mL x 1 via	Lyophilized powder, Tris buffer, ProClin300, TM antigen.
TM Quality Control level 2 (QC 2)	Dissolved to 1.0 mL x 1 via	Dissolved to 1.0 mL x 1 via	Lyophilized powder, Tris buffer, ProClin300, TM antigen.
Instruction for Use	1	1	N/A
Reagent Registration Card	1	1	N/A
Calibrator Barcode Sheet	1	1	N/A
Quality Control Sheet	1	1	N/A

Note: Refer to the target value list of the controls from Quality Control Sheet within the kit.

The Reagent Cartridge

The strip consists of 19 wells. The 1st well is the sample well and the last 19th well is the reading well for chemiluminescent signal. From the 2nd to the 18th well are all covered with a labeled, aluminum foil seal. The label comprises a barcode which mainly indicates the reagent assay information, reagent lot number and expiration date. All reagents components needed for the assay are ready-to-use and pre-dispensed in the cartridges. And please do not interchange integral component from different reagents or lots.

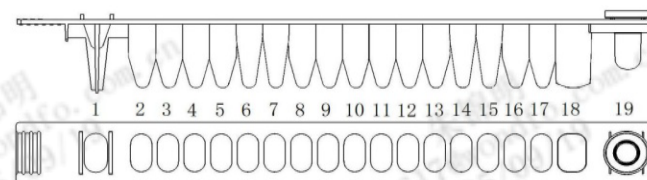

Description of the reagent cartridge

Well	Component	Content	Volume
5	Enzyme Label	ALP (alkaline phosphatase) labeled anti-TM antibody, MES Buffer, ProClin300.	50μL

7	Magnetic Particles	Anti-TM antibody coated magnetic particles, Tris buffer, ProClin300.	50μL
9	Sample diluent	Tris buffer; ProClin300.	30μL
2,3,4, 10,11,12	Wash Solutions	Tris buffer, ProClin300.	350μL
16	Substrate Solution	Disodium chlorophenyl)sulfanyl] methyl-9(10H)-acridinylidene) methyl phosphate.	200μL

Note: Well 1,6,8,13,14,15,17,18 are empty.

[MATERIALS AND DISPOSABLES REQUIRED BUT NOT PROVIDED]

- Sample diluent (manufactured by Tisenc), if needed.
- Pipette with disposable tip to dispense 200 μL.
- Powderless and disposable gloves.
- Disposable tips for ACCRE analyzer
- For other specific materials and disposables, please refer to the Instrument User Manual.
- Instruments of the ACCRE series analyzers.

[WARNINGS AND PRECAUTIONS]

- For *in vitro* diagnostic use only.
- For professional use only by qualified laboratory personnel.
- This kit contains products of animal origin. Certified knowledge of the origin and/or sanitary state of the animals does not totally guarantee the absence of transmissible pathogenic agents. It is therefore recommended that these products be treated as potentially infectious, and handled observing the usual safety precautions (do not ingest; do not inhale).
- Do not use reagents after the expiration date indicated on the box label.
- Do not use if the packaging of reagent cartridge is damaged or leaked.
- Do not use the reagents if the quantity is obviously insufficient.
- If any component adheres to inside wall of wells or aluminum film of the reagent cartridge, please gently switch the reagent cartridge to let the component back the wells.
- Do not reverse the reagent cartridge. Otherwise, the result will be unreliable.
- Package insert instructions must be carefully followed. Reliability of assay results cannot be guaranteed if there are any deviations from the instructions in this package insert.
- Use powderless gloves, as powder has been reported to cause false results for certain enzyme immunoassay tests.

[STORAGE]

- Store the Thrombomodulin (CLIA) upright at 2-8 °C, kit can be stored for 18 months from manufactured date. After the reagent cartridge is taken out from the refrigerator to equilibrate, the testing should be completed within 2 hours
- Store the calibrators and quality controls at 2-8 °C till expiry date. Calibrators and quality controls must be tested within 2 hours after being opened and re dissolved. The calibrators and/or controls will be stable for 30 days at -20 °C or below -20 °C, and without exceeding 1 freeze/thaw cycle.
- Keep away from sunlight.
- Do not freeze reagents cartridge.

[SAMPLES]

The sample volume for Thrombomodulin (CLIA) assay is 30μL, please note that at least 55μL of sample has to be pipetted manually into the reagent cartridge.

Sample type and collection

Human plasma/whole blood (Sodium citrate) are recommended.

Types of tubes:

-Plastic tube with sodium citrate.

It is recommended to validate collection tubes before using them as some may contain substances which interfere with test results.

Note: Blood sampling tube results may vary from one manufacturer to another depending on the materials and additives used.

It is the responsibility of each laboratory to validate the type of sample tube used and to follow the manufacturer's recommendations for use.

Sample preparation

The current revision of the WHO/DIL/LAB/99.1 document provides recommendations for the preparation of samples.

For use of the sample tubes, follow the tube manufacturer's instructions for use.

The pre-analytical step, including the preparation of blood samples, is an essential part of medical analyses. In accordance with Good Laboratory Practice, this step is performed under the responsibility of the laboratory manager.

Insufficient clot time can result in the formation of fibrin with micro-clots that are invisible to the naked eye. The presence of fibrin, red blood cells, or suspended particles can lead to erroneous results.

Samples containing suspended fibrin particles or erythrocyte stroma should be centrifuged before testing.

If there is a lipid layer over the sample after centrifuging, please transfer the clear sample into a new sample tube without transferring the lipid layer.

Severe hemolysis samples or heat-inactivated samples are not recommended for applying on the reagents.

Preparation of frozen-stored samples: after thawing, these samples must be thoroughly mixed before testing.

Mix using a vortex-type mixer. If necessary, clarify samples by centrifuging before testing.

Sample stability

Plasma samples can be stored at room temperature (15-30℃) for up to 8 hours or at 2-8 °C for up to 24 hours; if longer storage is required, freeze the plasma at -20 °C or below -20 °C for up to 1 months, without exceeding 1 freeze/thaw cycle.

Whole blood samples can be stored at room temperature (15-30℃) for up to 4 hours or at 2-8 °C for up to 24 hours and not undergo freeze/thaw cycles.

[INSTRUCTIONS FOR USE]

For complete instructions, see the Instrument User Manual.

When using the assay for the first time. Register the assay using the registration code in the reagent kit before any assay is used in the ACCRE analyzer for the first time. Calibration, QC and sample test are only available after assay was registered successfully.

When opening a new lot of reagents. Each new lot of reagent must be registered by using the registration code in Reagent Registration Card within the reagent kit.

Calibration

Calibration must be performed with the reagents and calibrators from Manufacturer.

Each bottle of lyophilized calibrator should be re-dissolved by distilled water, deionized water, or purified water to 1.0 mL before use. Mix it thoroughly and avoid foam formation. Keep dissolved calibrators upright for about 15

minutes to equilibrate to room temperature (15-30°C). Calibration is necessary under the following conditions:

- (1). New reagent lot number applied;
- (2). **Same lot of reagents used more than 4 weeks;**
- (3). QC out of control;
- (4). When necessary, for example: system maintenance or repair that may affect the analytic performance.

Test Procedure

1. Remove the kit from storage at 2-8 °C and take out the required number of reagent cartridges. Allow the reagent cartridges, specimens, calibrators, and/or controls to equilibrate to room temperature (15-30°C) prior to testing.
2. Use each assay strips for each sample, calibrator or control respectively.
3. Place the reagent cartridges into the reagent rack.
4. Manually pipette the Sample, Calibrator, or Control into the sample well of each reagent cartridge .

Please note that at least 55µL sample has to be pipetted manually into the reagent cartridge and 30µL sample is needed for each test.

- * Before pipetting: Mix the samples using a vortex-type mixer if necessary, and ensure that the samples and sample diluent (if applicable) DO NOT have any bubbles. The same for calibrators and controls.*
5. Put the sample rack into the reagent chamber of ACCRE analyzer.
 6. Place the assay tips into the reagent chamber of ACCRE analyzer.
 7. Input Sample ID, sample type, dilution etc. into the software as directed by USER MANUAL.
 8. Thrombomodulin (CLIA) assay will be performed automatically by the ACCRE analyzer.
 9. After assay is completed, remove the reagent rack from the ACCRE analyzer. Dispose the used reagent cartridge into an appropriate recipient.

Dilution

Samples with concentrations above 200 TU/mL can be diluted with sample diluent manually. The recommended dilution is 1:20. The concentration of the diluted sample must be >10.00TU/mL. After manual dilution, multiply the result by the dilution factor. ACCRE analyzer cannot perform automatic dilution for samples.

Quality control

Each bottle of lyophilized quality control should be re-dissolved by distilled water, deionized water, or purified water to 1.0mL before use. Mix it thoroughly and avoid foam formation. Keep dissolved quality controls upright for about 15 minutes to equilibrate to room temperature (15-30°C). Quality control should be performed in accordance with local regulations or requirements related to accreditation, as well as requirements defined in the laboratory's quality control procedure.

[RESULTS AND INTERPRETATION]

Calculation of Results

The results are automatically calculated by the ACCRE analyzer using stored calibration curve which generate by 2-point calibration, and calculated according to a predefined mathematical model, the results expressed in TU/mL.

Interpretation of Results

The detection range is from 1.00 TU/mL to 200.00 TU/mL. For samples with TM below the upper limit of detection, quantitative determination can be performed; if the sample is lower than the lower limit of detection, the reported result is <1.00 TU/mL; if the sample is higher than the upper limit of detection, the reported result is >200.00 TU/mL.

Results of Thrombomodulin (CLIA) assays were determined by testing plasma specimens from a study in clinical centers with group of totally 303 healthy individuals in China. The 2.5th - 95th percentile value is 3.84-13.49 TU/mL.

** Please note that results may differ between laboratories due to variations in population and test method. If necessary, each laboratory should establish its own reference range.*

In interpreting the results, patient's clinical condition including symptoms, diseases history, and other relevant data and information should be considered.

[LIMITATIONS]

- 1.For diagnostic purposes, results should be used in conjunction with other data; e.g., symptoms, results of other tests, clinical impressions, etc.
2. Please note that heterophile antibody in human blood can interfere the test result.
3. High concentration of HAMA can also cause a false positive or false negative result.
4. If the test results are inconsistent with clinical evidence, additional testing is suggested to confirm the result.

[PERFORMANCE CHARACTERISTICS]

Detection Limit

Limit of Detection(LoD) =1.00 TU/mL
The Limit of Detection was determined in accordance with the CLSI® EP17-A2 recommendations.

Measurement range

The measurement range is the range of values corresponding to the acceptable performance limits (precision and linearity).
The measurement range of the Thrombomodulin (CLIA) is 1.00 TU/mL~200.00 TU/mL.

Linearity

Linearity was evaluated according to CLSI® EP06-A recommendations. The Thrombomodulin (CLIA) assay is between 1.00 TU/mL~200.00 TU/mL. The absolute linear correlation coefficient (r) should be bigger than 0.9900.

Precision

The precision study was performed according to CLSI® EP05-A3 recommendations.
The within-lot precision CV% is less than 10%.

Interfering Substances

Potential interference substances were studied according to CLSI ® EP07-A3 recommendations. No significant interference was detected up to the maximum concentrations tested.

Substance	Concentration
Hemoglobin	≤ 450mg/dL
Bilirubin	≤ 20 mg/dL
Triglyceride	≤ 1400mg/dL
Rheumatoid factor	≤ 500 IU/mL

Accuracy

The relative deviation between the measured value and the calibration concentration is within the range of ±15%.

HAMA

Patient samples containing human anti-mouse antibodies (HAMA) may give falsely elevated or decreased values. Although HAMA-neutralizing agents are added, extremely high HAMA serum concentrations may occasionally

influence results.

[WASTE DISPOSAL]

Dispose used or unused reagents as well as any other contaminated disposable materials following procedures for infectious or potentially infectious products.
It is the responsibility of each laboratory to handle waste and effluents produced according to their nature and degree of hazardousness and to treat and dispose of them (or have them treated and disposed of) in accordance with any applicable regulations.

[INDEX OF SYMBOLS]

Symbol	Meaning	Symbol	Meaning	Symbol	Meaning
	Consult instructions for use		Contains sufficient for <n> tests		Date of manufacture
	In vitro diagnostic medical device		Use-by date		Do not re-use
	Temperature limit		Batch code		Catalogue number
	Keep away from sunlight		Keep dry		Manufacturer
	Authorized representative in the European Community/ European Union		Do not use if package is damaged and consult instructions for use		Contains biological material of animal origin
	Biological risks		Caution		CE marking
	Upright		Fragile, handle with care		Do not roll
	Control				

[REFERENCES]

- 1.Conway E M. Thrombomodulin and its role in inflammation[J]. Seminars in Immunopathology, 2012, 34(1):107-125.DOI:10.1007/s00281-011-0282-8.
- 2.Weisel JW, Nagaswami C, Young TA, Light DR. The shape of thrombomodulin and interactions with thrombin as determined by electron microscopy. J Biol Chem. 1996 Dec 6;271(49):31485-90. doi: 10.1074/jbc.271.49.31485. PMID: 8940162.
- 3.Komatsumoto S, Onoda A, Nara M. [Changes in the level of thrombomodulin with aging]. Nihon Ronen Igakkai Zasshi. 1996 Jul;33(7):512-7. Japanese. doi: 10.3143/geriatrics.33.512. PMID: 8890605.
- 4.Dohi Y, Ohashi M, Sugiyama M, et al. Circulati TU thrombomodulin levels are related to latent progression of atherosclerosis in hypertensive patients[J]. Hypertension Research, 2003, 26(6):479-483.
- 5.Blann A D, Mccollum C N. von Willebrand factor and soluble thrombomodulin as predictors of adverse events amoTU subjects with peripheral or coronary atherosclerosis[J]. Blood Coagulation & Fibrinolysis, 1999, 10(6):375-380.
- 6.Boscato Lm,Stuart MC.Heterophilic antibodies: a problem for all immunoassays. Clin Chem 1988;34:27-33.
7. Kricka L.Interferences in immunoassays - still a threat. Clin Chem 2000; 46:1037-1038.
- 8.Bjerner J, et al. Immunometric assay interference: incidence and prevention. Clin Chem 2002;48:613-621.

Shenzhen Tisenc Medical Devices Co., Ltd.
Address: 11F, 11G, Kechuang Building, Quanzhi Science and Innovation Park, Maozhoushan Industrial Park, Shajing Community, BaoAn District, Shenzhen, 518104, P.R.China
Phone: +86-400-830-8768 +86-0755-23225549
E-mail: sales@tisenc.com

EC REP

CMC Medical Devices & Drugs S.L.
C/ Horacio Lengo no 18, CP 29006, Málaga, Spain