



MAGLUMI® DHEA-S (CLIA)

INTENDED USE

The kit is an *in vitro* chemiluminescence immunoassay for the quantitative determination of DHEA-S in human serum and plasma using the MAGLUMI series Fully-auto chemiluminescence immunoassay analyzer and Biolumi series Integrated System, and the assay is used for an aid in the diagnosis and treatment of individuals with suspected or confirmed adrenal disease.

SUMMARY

DHEA and its sulphated form DHEA-S are the most abundant circulating steroid hormones. Sources of DHEA production are the adrenal gland, the gonads and the brain. More than 99% of the circulating DHEA is in the form of its sulphated ester DHEA-S and it was supposed that DHEAS may be a kind of reservoir that can be converted into the biologically active form of DHEA¹. In healthy women, the synthesis of DHEA and DHEA-S occurs exclusively in the adrenal cortex; in men, it is estimated that 5% of DHEA-S and 10% to 25% of the circulating DHEA are secreted by the testes². In both sexes, DHEA-S concentrations decline during the first year of life, are maintained for the next 5 to 6 years, then rise to mark the onset of adrenarche (the adrenal contribution to puberty); peak concentrations are reached between the ages of 20 and 30. By ages 70 to 80, DHEA-S concentrations decline to values 20% and 26% of peak concentrations in men and women, respectively². It has been suggested that the serum levels of dehydroepiandrosterone (DHEA) or dehydroepiandrosterone sulfate (DHEA-S) are markers of adrenal androgen production and, thus, could be useful to select patients for chronic glucocorticoid treatment³. 20–30 % of patients with PCOS may also present an increased adrenal production of dehydroepiandrosterone sulfate (DHEA-S) which after peripheral conversion to DHEA is further converted to androstendione and contributes indirectly to the increased extravarian testosterone formation⁴. Research found extremely low DHEAS levels in virtually all critically ill patients, in both septic shock and multiple trauma. DHEAS depletion was associated with a worse outcome and represents a prognostic marker⁵.

TEST PRINCIPLE

Competitive chemiluminescence immunoassay.

The sample, buffer, magnetic microbeads coated with DHEA-S antigen, ABEI labeled with anti-DHEA-S antibody are mixed thoroughly and incubated. DHEA-S present in the sample compete with DHEA-S antigen immobilized on the magnetic microbeads for binding anti-DHEA-S antibody labeled with ABEI, and form immuno-complexes. After precipitation in a magnetic field, the supernatant is decanted and then perform a wash cycle. Subsequently, the Starter 1+2 are added to initiate a chemiluminescent reaction. The light signal is measured by a photomultiplier as relative light units (RLUs), which is inversely proportional to the concentration of DHEA-S present in the sample.

REAGENTS

Kit Contents

Component	Description	100 tests/kit	50 tests/kit	30 tests/kit
Magnetic Microbeads	Magnetic microbeads coated with DHEA-S antigen (~8.00 µg/mL) in PBS buffer, NaNa ₃ (<0.1%).	2.5 mL	1.5 mL	1.0 mL
Calibrator Low	A low concentration of DHEA-S antigen, BSA, NaNa ₃ (<0.1%).	1.0 mL	1.0 mL	1.0 mL
Calibrator High	A high concentration of DHEA-S antigen, BSA, NaNa ₃ (<0.1%).	1.0 mL	1.0 mL	1.0 mL
Buffer	Tris-HCl buffer, NaNa ₃ (<0.1%).	6.5 mL	4.0 mL	3.0 mL
ABEI Label	ABEI labeled with anti-DHEA-S antibody (~0.125 µg/mL) in Tris-HCl buffer, NaNa ₃ (<0.1%).	7.5 mL	4.5 mL	3.3 mL
Control 1	A low concentration of DHEA-S antigen (100 µg/dL), BSA, NaNa ₃ (<0.1%).	1.0 mL	1.0 mL	1.0 mL
Control 2	A high concentration of DHEA-S antigen (500 µg/dL), BSA, NaNa ₃ (<0.1%).	1.0 mL	1.0 mL	1.0 mL

All reagents are provided ready-to-use.

The control barcode labels are provided.

Warnings and Precautions

- For *in vitro* diagnostic use.
- For professional use only.
- Exercise the normal precautions required for handling all laboratory reagents.
- Personal protective measures should be taken to prevent any part of the human body from contacting samples, reagents, and controls, and should comply with local operating requirements for the assay.
- A skillful technique and strict adherence to the package insert are necessary to obtain reliable results.
- Do not use kit beyond the expiration date indicated on the label.
- Do not interchange reagent components from different reagents or lots.
- Avoid foam formation in all reagents and sample types (specimens, calibrators and controls).
- All waste associated with biological samples, biological reagents and disposable materials used for the assay should be considered potentially infectious and should be disposed of in accordance with local guidelines.
- This product contains sodium azide. Sodium azide may react with lead or copper plumbing to form highly explosive metal azides. Immediately after disposal, flush with a large volume of water to prevent azide build-up. For additional information, see Safety Data Sheets available for professional user on request.

Note: If any serious incident has occurred in relation to the device, please report to Shenzhen New Industries Biomedical Engineering Co., Ltd. (Snibe) or our authorized representative and the competent authority of the Member State in which you are established.

Reagent Handling

- To avoid contamination, wear clean gloves when operating with a reagent kit and sample. When handling reagent kit, replace the gloves that have been in contact with samples, since introduction of samples will result in unreliable results.
- Do not use kit in malfunction conditions; e.g., the kit leaking at the sealing film or elsewhere, obviously turbid or precipitation is found in reagents (except for Magnetic Microbeads) or control value is out of the specified range repeatedly. When kit in malfunction conditions, please contact Snibe or our authorized distributor.
- To avoid evaporation of the liquid in the opened reagent kits in refrigerator, it is recommended that the opened reagent kits to be sealed with reagent seals contained within the packaging. The reagent seals are single use, and if more seals are needed, please contact Snibe or our authorized distributor.
- Over time, residual liquids may dry on the septum surface. These are typically dried salts and have no effect on assay efficacy.
- Use always the same analyzer for an opened reagent integral.
- For magnetic microbeads mixing instructions, refer to the Preparation of the Reagent section of this package insert.
- For further information about the reagent handing during system operation, please refer to Analyzer Operating Instructions.

Storage and Stability

- Do not freeze the integral reagents.
- Store the reagent kit upright to ensure complete availability of the magnetic microbeads.
- Protect from direct sunlight.

Stability of the Reagents	
Unopened at 2-8°C	until the stated expiration date

Opened at 2-8°C	6 weeks
On-board	4 weeks

Stability of Controls	
Unopened at 2-8°C	until the stated expiration date
Opened at 10-30°C	6 hours
Opened at 2-8°C	6 weeks
Frozen at -20°C	3 months
Frozen and thawed cycles	no more than 3 times

SPECIMEN COLLECTION AND PREPARATION

Specimen Types

Only the specimens listed below were tested and found acceptable.

Specimen Types	Collection Tubes
Serum	Tubes without additive/accessory, or tubes containing clot activator or clot activator with gel.
Plasma	K2-EDTA

- The sample types listed were tested with a selection of sample collection tubes that were commercially available at the time of testing, i.e. not all available tubes of all manufacturers were tested. Sample collection systems from various manufacturers may contain differing materials which could affect the test results in some cases. Follow tube manufacturers' instructions carefully when using collection tubes.

Specimen Conditions

- Do not use grossly hemolyzed/hyperlipidaemia specimens and specimens with obvious microbial contamination.
- Ensure that complete clot formation in serum specimens has taken place prior to centrifugation. Some serum specimens, especially those from patients receiving anticoagulant or thrombolytic therapy, may exhibit increased clotting time. If the serum specimen is centrifuged before a complete clotting, the presence of fibrin may cause erroneous results.
- Samples must be free of fibrin and other particulate matter.
- To prevent cross contamination, use of disposable pipettes or pipette tips is recommended.

Preparation for Analysis

- Inspect all specimens for foam. Remove foam with an applicator stick before analysis. Use a new applicator stick for each specimen to prevent cross contamination.
- Frozen specimens must be completely thawed before mixing. Mix thawed specimens thoroughly by low speed vortexing or by gently inverting. Visually inspect the specimens. If layering or stratification is observed, mix until specimens are visibly homogeneous. If specimens are not mixed thoroughly, inconsistent results may be obtained.
- Specimens should be free of fibrin, red blood cells, or other particulate matter. Such specimens may give reliable results and must be centrifuged prior to testing. Transfer clarified specimen to a sample cup or secondary tube for testing. For centrifuged specimens with a lipid layer, transfer only the clarified specimen and not the lipemic material.
- The sample volume required for a single determination of this assay is 10 µL.

Specimen Storage

Specimens removed from the separator, red blood cells or clot may be stored up to 8 hours at 10-30°C, or 8 days at 2-8°C, or 2 months frozen at -20°C. Frozen specimens subjected to up to 2 freeze/thaw cycles have been evaluated.

Specimen Shipping

- Package and label specimens in compliance with applicable local regulations covering the transport of clinical specimens and infectious substances.
- Do not exceed the storage limitations listed above.

Specimen Dilution

- Samples, DHEA-S concentrations above the analytical measuring interval, can be diluted with manual dilution procedure. The recommended dilution ratio is 1:10. The concentration of the diluted sample must be >150 µg/dL.
- For manual dilution, multiply the result by the dilution factor.
- Please choose applicable diluents or ask Snibe for advice before manual dilution.

PROCEDURE

Materials Provided

DHEA-S (CLIA) assay, control barcode labels.

Materials Required (But Not Provided)

- General laboratory equipment.
- Fully-auto chemiluminescence immunoassay analyzer Maglumi 600, Maglumi 800, Maglumi 1000, Maglumi 2000, Maglumi 2000 Plus, Maglumi 4000, Maglumi 4000 Plus, MAGLUMI X8, MAGLUMI X3, MAGLUMI X6, or Integrated System Biolumi 8000 and Biolumi CX8.
- Additional accessories of test required for the above analyzers include Reaction Module, Starter 1+2, Wash Concentrate, Light Check, Tip, and Reaction Cup. Specific accessories and accessories' specification for each model refer to corresponding Analyzer Operating Instructions.
- Please use accessories specified by Snibe to ensure the reliability of the test results.

Assay Procedure

Preparation of the Reagent

- Take the reagent kit out of the box and visually inspect the integral vials for leaking at the sealing film or elsewhere. If there is no leakage, please tear off the sealing film carefully.
- Open the reagent area door; hold the reagent handle to get the RFID label close to the RFID reader (for about 2s); the buzzer will beep; one beep sound indicates successful sensing.
- Keeping the reagent straight insert to the bottom along the blank reagent track.
- Observe whether the reagent information is displayed successfully in the software interface, otherwise repeat the above two steps.
- Resuspension of the magnetic microbeads takes place automatically when the kit is loaded successfully, ensuring the magnetic microbeads are totally resuspended homogenous prior to use.

Assay Calibration

- Select the assay to be calibrated and execute calibration operation in reagent area interface. For specific information on ordering calibrations, refer to the calibration section of Analyzer Operating Instructions.
- Execute recalibration according to the calibration interval required in this package insert.

Quality Control

- When new lot used, check or edit the quality control information.
- Scan the control barcode, choose corresponding quality control information and execute testing. For specific information on ordering quality controls, refer to the quality control section of the Analyzer Operating Instructions.

Sample Testing

- After successfully loading the sample, select the sample in interface and edit the assay for the sample to be tested and execute testing. For specific information on ordering patient specimens, refer to the sample ordering section of the Analyzer Operating Instructions.

To ensure proper test performance, strictly adhere to Analyzer Operating Instructions.

Calibration

Traceability: This method has been standardized against the Snibe internal reference standard.
Test of assay specific calibrators allows the detected relative light unit (RLU) values to adjust the master curve.
Recalibration is recommended as follows:

- Whenever a new lot of Reagent or Starter 1+2 is used.
- Every 28 days.
- The analyzer has been serviced.
- Control values lie outside the specified range.

Quality Control

Controls are recommended for the determination of quality control requirements for this assay and should be run in singlicate to monitor the assay performance. Refer to published guidelines for general quality control recommendations, for example Clinical and Laboratory Standards Institute (CLSI) Guideline C24 or other published guidelines⁶.

Quality control is recommended once per day of use, or in accordance with local regulations or accreditation requirements and your laboratory’s quality control procedures, quality control could be performed by running the DHEA-S assay:

- Whenever the kit is calibrated.
- Whenever a new lot of Starter 1+2 or Wash Concentrate is used.

Controls are only applicable with MAGLUMI and Biolumi system and only used matching with the same top seven LOT numbers of corresponding reagents. For each target value and range refer to the label.

The performance of other controls should be evaluated for compatibility with this assay before they are used. Appropriate value ranges should be established for all quality control materials used.

Control values must lie within the specified range, whenever one of the controls lies outside the specified range, calibration should be repeated and controls retested. If control values lie repeatedly outside the predefined ranges after successful calibration, patient results must not be reported and take the following actions:

- Verify that the materials are not expired.
- Verify that required maintenance was performed.
- Verify that the assay was performed according to the package insert.
- If necessary, contact Snibe or our authorized distributors for assistance.

If the controls in kit are not enough for use, please order DHEA-S (CLIA) Controls (REF: 160201259MT) from Snibe or our authorized distributors for more.

RESULTS

Calculation

The analyzer automatically calculates the DHEA-S concentration in each sample by means of a calibration curve which is generated by a 2-point calibration master curve procedure. The results are expressed in µg/dL. For further information please refer to the Analyzer Operating Instructions.
Conversion factor: µg/dL×0.02714=µmol/L;
µmol/L×36.846=µg/dL

Interpretation of Results

The expected range for the DHEA-S assay was obtained by testing 1729 apparently healthy individuals in China, gave the following expected value:

Test subjects		N	Mean (µg/dL)	5 th percentile (µg/dL)	95 th percentile (µg/dL)
Males Age (years)	11-14	64	108.387	16.4	243.7
	15-19	66	172.356	44.1	385.9
	20-24	128	366.414	237.4	539.9
	25-34	142	367.701	166.9	592.9
	35-44	130	295.990	139.1	488.4
	45-54	135	259.530	135.2	448.6
	55-64	133	154.475	48.2	362.8
	65-70	65	116.635	20.1	283.9
Females Age (years)	11-14	63	78.590	8.2	171.8
	15-19	67	217.485	60.2	494.6
	20-24	129	256.337	133.2	409.4
	25-34	135	287.696	94.8	513.7
	35-44	133	212.449	73.8	411.2
	45-54	136	153.726	55.2	283.9
	55-64	136	84.644	29.1	183.2
	65-70	67	52.713	33.2	79.1

Results may differ between laboratories due to variations in population and test method. It is recommended that each laboratory establish its own reference interval.

LIMITATIONS

- Results should be used in conjunction with patient’s medical history, clinical examination and other findings.
- If the DHEA-S results are inconsistent with clinical evidence, additional testing is needed to confirm the result.
- Specimens from patients who have received preparations of mouse monoclonal antibodies for diagnosis or therapy may contain human anti-mouse antibodies (HAMA). Such specimens may show either falsely elevated or depressed values when tested with assay kits which employ mouse monoclonal antibodies^{7,8}. Additional information may be required for diagnosis.
- Heterophilic antibodies in human serum can react with reagent immunoglobulins, interfering with *in vitro* immunoassays. Patients routinely exposed to animals or animal serum products can be prone to this interference and anomalous values may be observed⁹.
- Bacterial contamination of the specimens may affect the test results.

SPECIFIC PERFORMANCE CHARACTERISTICS

Representative performance data are provided in this section. Results obtained in individual laboratories may vary.

Precision

Precision was determined using the assay, samples and controls in a protocol (EP05-A3) of the CLSI (Clinical and Laboratory Standards Institute): duplicates at two independent runs per day for 5 days at three different sites using three lots of reagent kits (n = 180). The following results were obtained:

Sample	Mean (µg/dL) (n=180)	Within-Run		Between-Run		Reproducibility	
		SD (µg/dL)	%CV	SD (µg/dL)	%CV	SD (µg/dL)	%CV
Serum Pool 1	10.098	0.382	3.78	0.256	2.54	0.606	6.00
Serum Pool 2	100.381	3.762	3.75	0.891	0.89	4.958	4.94
Serum Pool 3	554.309	18.037	3.25	7.576	1.37	26.592	4.80
Plasma Pool 1	9.885	0.400	4.05	0.253	2.56	0.589	5.96
Plasma Pool 2	98.767	3.609	3.65	1.661	1.68	6.608	6.69
Plasma Pool 3	544.451	17.659	3.24	9.301	1.71	25.904	4.76
Control 1	99.551	4.200	4.22	1.909	1.92	5.628	5.65
Control 2	508.147	19.414	3.82	9.770	1.92	24.500	4.82

Linear Range

4.00-1500 µg/dL (defined by the Limit of Quantitation and the maximum of the master curve).

Reportable Interval

2.00-15000 µg/dL (defined by the Limit of Detection and the maximum of the master curve×Recommended Dilution Ratio).

Analytical Sensitivity

Limit of Blank (LoB) =0.100 µg/dL.
Limit of Detection (LoD) =2.00 µg/dL.
Limit of Quantitation (LoQ) =4.00 µg/dL.

Analytical Specificity

Interference

Interference was determined using the assay, three samples containing different concentrations of analyte were spiked with potential endogenous and exogenous interferences in a protocol (EP7-A2) of the CLSI. The measurement deviation of the interference substance is within ±10%. The following results were obtained:

Interference	No interference up to	Interference	No interference up to
Bilirubin	20 mg/dL	Rheumatoid factor	1500 IU/mL
Hemoglobin	500 mg/dL	ANA	398 AU/mL
Intralipid	2000 mg/dL	Biotin	0.5 mg/dL
HAMA	40 ng/mL	Glucuronic acid	2000 µg/dL

Cross-Reactivity

Cross-reactivity was determined using the assay, three samples containing different concentrations of analyte were spiked with potential cross-reactants in a protocol (EP7-A2) of the CLSI. The measurement deviation of the interference substance is within ±10%. The following results were obtained:

Cross-reactant	No interference up to	Cross-reactant	No interference up to
Dehydroepiandrosterone	4000 µg/dL	Ketosteroid	2000 µg/dL
Cortisol	10000 µg/dL	Estrisol	5000 µg/dL
Aldosterone	5000 µg/dL	Oestrone	5000 µg/dL
Estradiol	5000 µg/dL	19-hydroxyandrostenedione	1000 µg/dL
Testosterone	2000 µg/dL	Progesterone	5000 µg/dL
5α-dihydrotestosterone	5000 µg/dL	Androsterone sulfate	5000 µg/dL
Androstenedione	1000 µg/dL	Estrone-3-sulfate	5000 µg/dL
DHEA glucuronic acid	5000 µg/dL		

Method Comparison

A comparison of the DHEA-S assay with a commercially available immunoassay, gave the following correlations (µg/dL):

Number of samples measured: 335

Passing-Bablok: y=1.0049x-0.4180, r=0.977.

The clinical specimen concentrations were between 5.9 and 1445.7 µg/dL.

REFERENCES

1. Oberbeck R, Kobbe P. Dehydroepiandrosterone (DHEA): a steroid with multiple effects. Is there any possible option in the treatment of critical illness? [J]. Current medicinal chemistry, 2010, 17(11): 1039-1047.
2. Kroboth P D, Salek F S, Pittenger A L, et al. DHEA and DHEA-S: A Review [J]. The Journal of Clinical Pharmacology, 1999, 39(4): 327-348.
3. STEINBERGER E, SMITH K D, RODRIGUEZ-RIGAU L J. Testosterone, dehydroepiandrosterone, and dehydroepiandrosterone sulfate in hyperandrogenic women [J]. The Journal of Clinical Endocrinology & Metabolism, 1984, 59(3): 471-477.
4. Rachoń D. Differential diagnosis of hyperandrogenism in women with polycystic ovary syndrome [J]. Experimental and clinical endocrinology & diabetes, 2012, 120(04): 205-209.
5. Beishuizen A, Thijs L G, Vermes I. Decreased levels of dehydroepiandrosterone sulphate in severe critical illness: a sign of exhausted adrenal reserve? [J]. Critical Care, 2002, 6(5): 434-438.
6. CLSI. Statistical Quality Control for Quantitative Measurement Procedures: Principles and Definitions. 4th ed. CLSI guideline C24. Wayne, PA: Clinical and Laboratory Standards Institute; 2016.
7. Robert W. Schroff, Kenneth A. Foon, Shannon M. Beatty, et al. Human Anti-Murine Immunoglobulin Responses in Patients Receiving Monoclonal Antibody Therapy [J]. Cancer Research, 1985, 45(2):879-885.
8. Primus F J, Kelley E A, Hansen H J, et al. "Sandwich"-type immunoassay of carcinoembryonic antigen in patients receiving murine monoclonal antibodies for diagnosis and therapy [J]. Clinical Chemistry, 1988, 34(2):261-264.
9. Boscato L M, Stuart M C. Heterophilic antibodies: a problem for all immunoassays [J]. Clinical Chemistry, 1988,34(1):27-33.

SYMBOLS EXPLANATIONS

	Consult instructions for use		Manufacturer
	Temperature limit (Store at 2-8 °C)		Use-by date
	Contains sufficient for<n> tests		Keep away from sunlight
	This way up		Authorized representative in the European Community
	In vitro diagnostic medical device		Kit component
	Catalogue number		Batch code
	CE marking with notified body ID number		

MAGLUMI® and Biolumi® are trademarks of Snibe. All other product names and trademarks are the property of their respective owners.

Shenzhen New Industries Biomedical Engineering Co., Ltd.
No.23, Jinxiu East Road, Pingshan District, 518122 Shenzhen, P.R. China
Tel: +86-755-21536601 Fax: +86-755-28292740

Shanghai International Holding Corp. GmbH (Europe)
Eiffestrasse 80, 20537 Hamburg, Germany
Tel: +49-40-2513175 Fax: +49-40-255726