



MAGLUMI® Thrombin-Antithrombin III Complex (CLIA)

INTENDED USE

The kit is an *in vitro* chemiluminescence immunoassay for the quantitative determination of Thrombin-Antithrombin III Complex (TAT) in human plasma using the MAGLUMI series Fully-auto chemiluminescence immunoassay analyzer and Biolumi series Integrated System, and the assay is used for an aid in the diagnosis of individuals with suspected or confirmed thrombotic diseases.

SUMMARY

Thrombin-Antithrombin III Complex (TAT) is a molecular complex composed of thrombin and antithrombin III (ATIII) in a 1:1 ratio. Thrombin is composed of a light A-chain (36 residues) and a heavy B-chain (259 residues) linked covalently by an inter-chain disulfide bond with a molecular weight of 37 kDa, which can convert fibrinogen into insoluble fibrin and form thrombus^{1,2}. ATIII is a serine protease inhibitor, which is mainly synthesized in liver cells and has a relative molecular weight of 58.2 kDa³. During the reaction in the blood coagulation cascade, factor Xa cleaves prothrombin, which results in the generation of thrombin. This serine protease can rapidly be neutralized by ATIII, with the formation of inactive TAT. TAT was considered to reflect the formation of thrombin attendant on the activation of coagulation factors and to serve as a sensitive parameter of activated coagulation in circulating blood.⁴

The increase of TAT concentration reflects the activation of the coagulation system and the generation of thrombin, which can be used in the auxiliary diagnosis of thrombotic diseases. Increased TAT indexes are commonly seen in disseminate intravascular coagulation (DIC) caused by severe infection, severe trauma, obstetric complications and other basic diseases^{5,6}. TAT can be used for early classification of DIC^{7,8}, monitoring postoperative changes in coagulation function, monitoring the efficacy of thrombolysis⁹, warning the risk of thrombosis in patients with fracture, and screening arteriovenous thrombosis¹⁰. Abnormal TAT is also common in acute myocardial infarction, cerebral infarction, pulmonary embolism, vasculitis, etc¹¹⁻¹⁴.

TEST PRINCIPLE

Sandwich chemiluminescence immunoassay.

The sample, buffer, magnetic microbeads coated with anti-Thrombin monoclonal antibody are mixed thoroughly, incubating and performing a wash cycle after a precipitation in a magnetic field. ABEI labeled with anti-Anithrombin III monoclonal antibody are then added, reacting to form sandwich complexes and incubating. After precipitation in a magnetic field, the supernatant is decanted and then a wash cycle is performed. Subsequently, the Starter 1+2 are added to initiate a chemiluminescent reaction. The light signal is measured by a photomultiplier as relative light units (RLUs), which is proportional to the concentration of TAT present in the sample.

REAGENTS

Kit Contents

Component	Description	100 tests/kit	50 tests/kit	30 tests/kit
Magnetic Microbeads	Magnetic microbeads coated with anti-Thrombin monoclonal antibody (~10.0 µg/mL) in PBS buffer, NaNa ₃ (<0.1%).	2.5 mL	1.5 mL	1.0 mL
Calibrator Low	A low concentration of TAT antigen in PBS buffer, NaNa ₃ (<0.1%).	1.0 mL	1.0 mL	1.0 mL
Calibrator High	A high concentration of TAT antigen in PBS buffer, NaNa ₃ (<0.1%).	1.0 mL	1.0 mL	1.0 mL
Buffer	PBS buffer, NaNa ₃ (<0.1%).	12.5 mL	7.0 mL	4.8 mL
ABEI Label	ABEI labeled with anti- Antithrombin III monoclonal antibody (~125 ng/mL) in Tris-HCl buffer, NaNa ₃ (<0.1%).	22.5 mL	12.0 mL	7.8 mL
Diluent	PBS buffer, NaNa ₃ (<0.1%).	5.5 mL	3.5 mL	3.5 mL
Control 1	A low concentration of TAT antigen (5.00 ng/mL) in PBS buffer, NaNa ₃ (<0.1%).	1.0 mL	1.0 mL	1.0 mL
Control 2	A high concentration of TAT antigen (50.0 ng/mL) in PBS buffer, NaNa ₃ (<0.1%).	1.0 mL	1.0 mL	1.0 mL
All reagents are provided ready-to-use.				

Warnings and Precautions

- For *in vitro* diagnostic use.
- For professional use only.
- Exercise the normal precautions required for handling all laboratory reagents.
- Personal protective measures should be taken to prevent any part of the human body from contacting samples, reagents, and controls, and should comply with local operating requirements for the assay.
- A skillful technique and strict adherence to the package insert are necessary to obtain reliable results.
- Do not use kit beyond the expiration date indicated on the label.
- Do not interchange reagent components from different reagents or lots.
- Avoid foam formation in all reagents and sample types (specimens, calibrators and controls).
- All waste associated with biological samples, biological reagents and disposable materials used for the assay should be considered potentially infectious and should be disposed of in accordance with local guidelines.
- This product contains sodium azide. Sodium azide may react with lead or copper plumbing to form highly explosive metal azides. Immediately after disposal, flush with a large volume of water to prevent azide build-up. For additional information, see Safety Data Sheets available for professional user on request.

Note: If any serious incident has occurred in relation to the device, please report to Shenzhen New Industries Biomedical Engineering Co., Ltd. (Snibe) or our authorized representative and the competency authority of the Member State in which you are established.

Reagent Handling

- To avoid contamination, wear clean gloves when operating with a reagent kit and sample. When handling reagent kit, replace the gloves that have been in contact with samples, since introduction of samples will result in unreliable results.
- Do not use kit in malfunction conditions; e.g., the kit leaking at the sealing film or elsewhere, obviously turbid or precipitation is found in reagents (except for Magnetic Microbeads) or control value is out of the specified range repeatedly. When kit in malfunction conditions, please contact Snibe or our authorized distributor.
- To avoid evaporation of the liquid in the opened reagent kits in refrigerator, it is recommended that the opened reagent kits to be sealed with reagent seals contained within the packaging. The reagent seals are single use, and if more seals are needed, please contact Snibe or our authorized distributor.
- Over time, residual liquids may dry on the septum surface. These are typically dried salts and have no effect on assay efficacy.
- Use always the same analyzer for an opened reagent integral.
- For magnetic microbeads mixing instructions, refer to the Preparation of the Reagent section of this package insert.
- For further information about the reagent handling during system operation, please refer to Analyzer Operating Instructions.

Storage and Stability

- Do not freeze the integral reagents.
- Store the reagent kit upright to ensure complete availability of the magnetic microbeads.
- Protect from direct sunlight.

Stability of the Reagents	
Unopened at 2-8°C	until the stated expiration date
Opened at 2-8°C	6 weeks
On-board	4 weeks



0123



130218001M: 100 tests/kit
130618001M: 50 tests/kit
130718001M: 30 tests/kit

Stability of Controls	
Unopened at 2-8°C	until the stated expiration date
Opened at 10-30°C	6 hours
Opened at 2-8°C	6 weeks
Frozen at -20°C	3 months
Frozen and thawed cycles	no more than 3 times

SPECIMEN COLLECTION AND PREPARATION

Specimen Types

Only the specimens listed below were tested and found acceptable.

Specimen Types	Collection Tubes
Plasma	Sodium citrate (1:9)

- The sample types listed were tested with a selection of sample collection tubes that were commercially available at the time of testing, i.e. not all available tubes of all manufacturers were tested. Sample collection systems from various manufacturers may contain differing materials which could affect the test results in some cases. Follow tube manufacturers' instructions carefully when using collection tubes.

Specimen Conditions

- Do not use grossly hemolyzed/hyperlipidaemia specimens and specimens with obvious microbial contamination.
- Samples must be free of fibrin and other particulate matter.
- To prevent cross contamination, use of disposable pipettes or pipette tips are recommended.

Preparation for Analysis

- Inspect all specimens for foam. Remove foam with an applicator stick before analysis. Use a new applicator stick for each specimen to prevent cross contamination.
- Frozen specimens must be completely thawed before mixing. Mix thawed specimens thoroughly by low speed vortexing or by gently inverting. Visually inspect the specimens. If layering or stratification is observed, mix until specimens are visibly homogeneous. If specimens are not mixed thoroughly, inconsistent results may be obtained.
- Specimens should be free of fibrin, red blood cells, or other particulate matter. Such specimens may give reliable results and must be centrifuged prior to testing. Transfer clarified specimen to a sample cup or secondary tube for testing. For centrifuged specimens with a lipid layer, transfer only the clarified specimen and not the lipemic material.
- The sample volume required for a single determination of this assay is 40 µL.

Specimen Storage

Specimens removed from the separator, red blood cells or clot may be stored up to 4 hours at 10-30°C or 24 hours at 2-8°C, or 12 months frozen at -20°C. Frozen specimens subjected to up to 1 freeze/thaw cycle have been evaluated.

Specimen Shipping

- Package and label specimens in compliance with applicable local regulations covering the transport of clinical specimens and infectious substances.
- Do not exceed the storage limitations listed above.

Specimen Dilution

- Samples, with TAT concentrations above the analytical measuring interval, can be diluted with Diluent either by following automated dilution protocol or manual dilution procedure. The recommended dilution ratio is 1:10. The concentration of the diluted sample must be >12 ng/mL.
- For manual dilution, multiply the result by the dilution factor. For dilution by the analyzers, the analyzer software automatically takes the dilution into account when calculating the sample concentration.

PROCEDURE

Materials Provided

Thrombin-Antithrombin III Complex (CLIA) assay, control barcode labels.

Materials Required (But Not Provided)

- General laboratory equipment.
- Fully-auto chemiluminescence immunoassay analyzer Maglumi 600, Maglumi 800, Maglumi 1000, Maglumi 2000, Maglumi 2000 Plus, Maglumi 4000, Maglumi 4000 Plus, MAGLUMI X3, MAGLUMI X6, MAGLUMI X8, MAGLUMI X8i, or Integrated System Biolumi 8000 and Biolumi CX8.
- Additional accessories of test required for the above analyzers include Reaction Module, Starter 1+2, Wash Concentrate, Light Check, Tip, and Reaction Cup. Specific accessories and accessories' specification for each model refer to corresponding Analyzer Operating Instructions.
- Please use accessories specified by Snibe to ensure the reliability of the test results.

Assay Procedure

Preparation of the Reagent

- Take the reagent kit out of the box and visually inspect the integral vials for leaking at the sealing film or elsewhere. If there is no leakage, please tear off the sealing film carefully.
- Open the reagent area door; hold the reagent handle to get the RFID label close to the RFID reader (for about 2s); the buzzer will beep; one beep sound indicates successful sensing.
- Keeping the reagent straight insert to the bottom along the blank reagent track.
- Observe whether the reagent information is displayed successfully in the software interface, otherwise repeat the above two steps.
- Resuspension of the magnetic microbeads takes place automatically when the kit is loaded successfully, ensuring the magnetic microbeads are totally resuspended homogenous prior to use.

Assay Calibration

- Select the assay to be calibrated and execute calibration operation in reagent area interface. For specific information on ordering calibrations, refer to the calibration section of Analyzer Operating Instructions.
- Execute recalibration according to the calibration interval required in this package insert.

Quality Control

- When new lot used, check or edit the quality control information.
- Scan the control barcode, choose corresponding quality control information and execute testing. For specific information on ordering quality controls, refer to the quality control section of the Analyzer Operating Instructions.

Sample Testing

- After successfully loading the sample, select the sample in interface and edit the assay for the sample to be tested and execute testing. For specific information on ordering patient specimens, refer to the sample ordering section of the Analyzer Operating Instructions.

To ensure proper test performance, strictly adhere to Analyzer Operating Instructions.

Calibration

Traceability: This method has been standardized against the Snibe internal reference standard.

Test of assay specific calibrators allows the detected relative light unit (RLU) values to adjust the master curve.

Recalibration is recommended as follows:

- Whenever a new lot of Reagent or Starter 1+2 is used.
- Every 7 days.
- The analyzer has been serviced.
- Control values lie outside the specified range.

Quality Control

Controls are recommended for the determination of quality control requirements for this assay and should be run in singlicate to monitor the assay performance. Refer to published guidelines for general quality control recommendations, for example Clinical and Laboratory Standards Institute (CLSI) Guideline C24 or other

published guidelines¹⁵. Quality control is recommended once per day of use, or in accordance with local regulations or accreditation requirements and your laboratory's quality control procedures, quality control could be performed by running the TAT assay:

- Whenever the kit is calibrated.
- Whenever a new lot of Starter 1+2 or Wash Concentrate is used.

Controls are only applicable with MAGLUMI and Biolumi systems and only used matching with the same top seven LOT numbers of corresponding reagents. For each target value and range refer to the label.

The performance of other controls should be evaluated for compatibility with this assay before they are used. Appropriate value ranges should be established for all quality control materials used.

Control values must lie within the specified range, whenever one of the controls lies outside the specified range, calibration should be repeated and controls retested. If control values lie repeatedly outside the predefined ranges after successful calibration, patient results must not be reported and take the following actions:

- Verify that the materials are not expired.
- Verify that required maintenance was performed.
- Verify that the assay was performed according to the package insert.
- If necessary, contact Snibe or our authorized distributors for assistance.

If the controls in kit are not enough for use, please order Thrombin-Antithrombin III Complex (CLIA) Controls (REF: 160201234MT) from Snibe or our authorized distributors for more.

■ RESULTS

Calculation
The analyzer automatically calculates the TAT concentration in each sample by means of a calibration curve which is generated by a 2-point calibration master curve procedure. The results are expressed in ng/mL. For further information please refer to the Analyzer Operating Instructions.

Interpretation of Results
The expected range for the TAT assay was obtained by testing 571 apparently healthy individuals in China, gave the following expected value:
≤4 ng/mL (95th percentile).
Results may differ between laboratories due to variations in population and test method. It is recommended that each laboratory establish its own reference interval.

■ LIMITATIONS

- Results should be used in conjunction with patient's medical history, clinical examination and other findings.
- If the TAT results are inconsistent with clinical evidence, additional testing is needed to confirm the result.
- Specimens from patients who have received preparations of mouse monoclonal antibodies for diagnosis or therapy may contain human anti-mouse antibodies (HAMA). Such specimens may show either falsely elevated or depressed values when tested with assay kits which employ mouse monoclonal antibodies^{16,17}. Additional information may be required for diagnosis.
- Heterophilic antibodies in human serum can react with reagent immunoglobulins, interfering with *in vitro* immunoassays. Patients routinely exposed to animals or animal serum products can be prone to this interference and anomalous values may be observed¹⁸.
- Bacterial contamination or heat inactivation of the specimens may affect the test results.

■ SPECIFIC PERFORMANCE CHARACTERISTICS

Representative performance data are provided in this section. Results obtained in individual laboratories may vary.

Precision

Precision was determined using the assay, samples and controls in a protocol (EP05-A3) of the CLSI (Clinical and Laboratory Standards Institute): duplicates at two independent runs per day for 5 days at three different sites using three lots of reagent kits (n = 180). The following results were obtained:

Sample	Mean (ng/mL) (n=180)	Within-Run		Between-Run		Reproducibility	
		SD (ng/mL)	%CV	SD (ng/mL)	%CV	SD (ng/mL)	%CV
Plasma Pool 1	3.808	0.105	2.76	0.066	1.73	0.136	3.57
Plasma Pool 2	20.667	0.478	2.31	0.179	0.87	0.734	3.55
Plasma Pool 3	101.937	2.012	1.97	1.380	1.35	3.008	2.95
Control 1	5.001	0.160	3.20	0.093	1.86	0.221	4.42
Control 2	51.127	1.306	2.55	0.553	1.08	2.155	4.21

Linear Range
0.400-120 ng/mL (defined by the Limit of Quantitation and the maximum of the master curve).

Reportable Interval
0.300-1200 ng/mL (defined by the Limit of Detection and the maximum of the master curve×Recommended Dilution Ratio).

Analytical Sensitivity
Limit of Blank (LoB) =0.150 ng/mL.
Limit of Detection (LoD) =0.300 ng/mL.
Limit of Quantitation (LoQ) =0.400 ng/mL.

Analytical Specificity

Interference
Interference was determined using the assay, three samples containing different concentrations of analyte were spiked with potential endogenous and exogenous interferents in a protocol (EP7-A2) of the CLSI. The measurement deviation of the interference substance is within ±10%. The following results were obtained:

Interference	No interference up to
Bilirubin	20 mg/dL
Hemoglobin	500 mg/dL
Intralipid	1000 mg/dL
HAMA	40 ng/mL
Rheumatoid factor	1500 IU/mL
ANA	398 AU/mL

Cross-Reactivity

Cross-reactivity was determined using the assay, three samples containing different concentrations of analyte were spiked with potential cross-reactant in a protocol (EP7-A2) of the CLSI. The measurement deviation of the interference substance is within ±10%. The following results were obtained:

Cross-reactant	No interference up to
Thrombinogen	0.2 mg/mL
Antithrombin III	0.4 mg/mL

High-Dose Hook

No high-dose hook effect was seen for TAT concentrations up to 2000 ng/mL.

Method Comparison

A comparison of the TAT assay with a commercially available immunoassay, gave the following correlations (ng/mL):
Number of samples measured: 229
Passing-Bablok: y=1.0045x-0.0098, r=0.982.
The clinical specimen concentrations were between 0.415 and 117.192 ng/mL.

■ REFERENCES

1. Chen Q, Shou W, Wu W, et al. Performance evaluation of thrombomodulin, thrombin-antithrombin complex, plasmin-α2-antiplasmin complex, and t - PA: PAI-1 complex[J]. Journal of Clinical Laboratory Analysis, 2019, 33(4):e22913.

2. Rajesh, Singh, R, et al. Structural stability of human a-thrombin studied by disulfide reduction and scrambling[J]. BIOCHIMICA ET BIOPHYSICA ACTA, 2003.
3. Patnaik M M, Moli S. Inherited antithrombin deficiency: a review[J]. Haemophilia, 2010, 14(6):1229-1239.
4. Deguchi K, Noguchi M, Yuwasaki E, et al. Dynamic fluctuations in blood of thrombin/antithrombin III complex (TAT)[J]. American Journal of Hematology, 2010, 38(2):86-89.
5. Levi M, ten Cate H. Disseminated intravascular coagulation[J]. N Engl J Med, 1999, 341: 586-592.
6. Heng, Mei, Ying, et al. Evaluation the combined diagnostic value of TAT, PIC, tPAIC, and sTM in disseminated intravascular coagulation: A multi-center prospective observational study[J]. Thrombosis research, 2018, 173:20-26.
7. Koyama K, Madoiwa S, Nunomiya S, et al. Combination of thrombin-antithrombin complex, plasminogen activator inhibitor-1, and protein C activity for early identification of severe coagulopathy in initial phase of sepsis: a prospective observational study[J]. Critical Care, 2014, 18(1):R13.
8. Taylor F B J, Toh C H, Hoots W K, et al. Towards definition, clinical and laboratory criteria, and a scoring system for disseminated intravascular coagulation.[J]. Thromb Haemost, 2001, 85(5):1327-1330.
9. Andreotti F, Hackett D R, Davies G J, et al. Low thrombin-antithrombin III complex levels predict successful thrombolysis in acute myocardial infarction[J]. Journal of the American College of Cardiology, 1991, 17(2):A347.
10. Sang, Yang, Lee, et al. Thrombin-antithrombin III complex tests[J]. Journal of Orthopaedic Surgery, 2017, 25(1).
11. Szczeklik A, Dropinski J, Radwan J, et al. Persistent generation of thrombin after acute myocardial infarction[J]. Arteriosclerosis & Thrombosis A Journal of Vascular Biology, 1992, 12(5):548.
12. Kataoka S, Hirose G, Hori A, et al. Activation of thrombosis and fibrinolysis following brain infarction[J]. Journal of the Neurological Sciences, 2000, 181(1-2):82-88.
13. F Carmassi, Morale M, De Negri, et al. Thrombin-antithrombin III complexes as an additional diagnostic aid in pulmonary embolism.[J]. Haemostasis, 1996, 26(1):16-22.
14. Hergesell O, Andrassy K, Nawroth P, Elevated levels of markers of endothelial cell damage and markers of activated coagulation in patients with systemic necrotizing vasculitis[J]. Thromb Haemost, 1996, 76.
15. CLSI. Statistical Quality Control for Quantitative Measurement Procedures: Principles and Definitions. 4th ed. CLSI guideline C24. Wayne, PA: Clinical and Laboratory Standards Institute; 2016.
16. Primus F J, Kelley E A, Hansen H J, et al. "Sandwich"-type immunoassay of carcinoembryonic antigen in patients receiving murine monoclonal antibodies for diagnosis and therapy[J]. Clinical Chemistry, 1988, 34(2):261-4.
17. Robert W. Schroff, Kenneth A. Foon, Shannon M. Beatty, et al. Human anti-murine immunoglobulin responses in patients receiving monoclonal antibody therapy.[J]. Cancer Research, 1985, 45(2):879-885.
18. Boscatto L M, Stuart M C. Heterophilic antibodies: a problem for all immunoassays[J]. Clinical Chemistry, 1988, 34(1):27-33.

■ SYMBOLS EXPLANATIONS

	Consult instructions for use		Manufacturer
	Temperature limit (Store at 2-8°C)		Use-by date
	Contains sufficient for<n> tests		Keep away from sunlight
	This way up		Authorized representative in the European Community
	In vitro diagnostic medical device		Kit component
	Catalogue number		Batch code
	CE marking with notified body ID number		

MAGLUMI® and Biolumi® are trademarks of Snibe. All other product names and trademarks are the property of their respective owners.

Summary of safety and performance is available at Eudamed.

	Shenzhen New Industries Biomedical Engineering Co., Ltd. No.23, Jinxiu East Road, Pingshan District, 518122 Shenzhen, P.R. China Tel: +86-755-21536601 Fax: +86-755-28292740
	Shanghai International Holding Corp. GmbH (Europe) Eiffestrasse 80, 20537 Hamburg, Germany Tel: +49-40-2513175 Fax: +49-40-255726